HUMANTECHNIK

PL-51

DE

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Seite 2

Telefon-Hörverstärker

EN

User manual

Page 8

Telephone amplifier

FR

Mode d'emploi

Page 14

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NL

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Pagina 20

Telefoon-hoornverstärker

IT

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Pagina 26

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
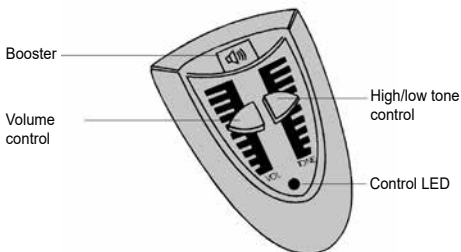
Operating elements

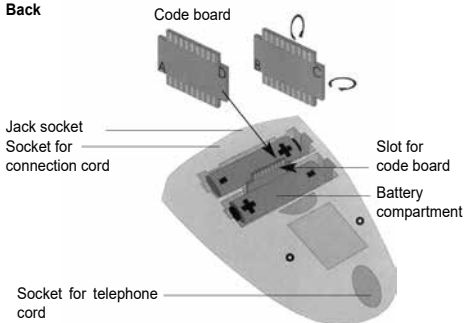Back

Congratulations on purchasing your *telephone amplifier PL-51*. We hope it will give you many years of reassuring operation. Please read this manual carefully. It will tell you how to set up the system correctly and explain all the system's features to you.

Standard components

Please check if all following components are included:

- *Telephone amplifier PL-51*
- Connection cord
- 2 x 1,5 V mignon (AA) batteries
- User manual
- Warranty card

Should any parts be missing, please contact your dealer or the manufacturer directly.

Setting up the system

Please follow these steps before using the telephone amplifier:

1. Telephone receiver cord

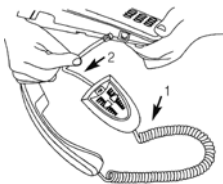
Unplug the telephone cord from the telephone connection socket. Remove the receiver cord from the cable guide on the underside of your telephone by pushing the locking clip towards the body of the plug and pulling it out together with the cord.

This plug of the receiver cord (1) is plugged into the socket on the small side of the PL-51. Make sure the plug catches completely.

2. Connection cord

Plug one end of the supplied connection cord (2) into the wide side of the PL-51 and plug the other end into the receiver socket on the underside of the telephone. Position the cord in the empty cable guide. Finally reconnect the telephone cord to the telephone connection socket.

Your telephone is now ready to operate again.

Switch on

The hearing amplifier automatically switches on when the receiver of the telephone is picked up. The green light on top of the unit lights up to indicate that the amplifier is active.

Setting the volume/tone

Adjust the volume control to low before picking up the telephone and set the high/low tone control to medium.

Now the volume can be adjusted to your needs with the volume control.

The high/low tone control can be used to alter the tone by emphasising high or low tones. Furthermore the output volume can be doubled by pressing the amplifier booster button.

The set volume is increased when the amplifier button is pressed down and held. The button must be disengaged while talking.

Switch off

The hearing amplifier switches off automatically approx. 30 seconds after the receiver is placed back on the hook. The green light goes out.

Connecting accessories

The following optional accessories can be connected to the 3.5 mm jack socket of the *PL-51*:

- Small headphones.
- Hearing aid users can connect a neck loop to the jack socket. The hearing aids have to be switched to „T“ or „MT“.

The volume can be adjusted with the volume control of the *PL-51*.

- Hearing aids with audio shoe can be connected to the *PL-51* with the DAI cable monaural (for people wearing one hearing aid) or binaural (for people wearing hearing aids in both ears)

Please note that the loudspeaker of the handset is turned off when the jack socket is used. This serves to prevent feedback.

Accessories

- Underchin headphones
- Earphones
- Light-weight headphones
- Induction hooks monaural
- Induction hooks binaural
- DAI cable monaural
- DAI cable binaural

The *PL-51* can be connected to the following units via the jack socket:

- Infrared transmission systems
- Radio transmitting systems
- Inductive loop systems

Power supply of the PL-51

The *PL-51* is operated with 2 batteries (type AA, LR6), which last up to 2 years. Rechargeable batteries of the same type can also be used.

Please note that when using rechargeable batteries the operating time is shortened. Unplug the telephone cord from the telephone connection socket before changing the batteries. The battery compartment is on the under side of the *PL-51*. Pressing the clip against the cover opens the battery compartment. The cover can now be lifted off. Insert 2 batteries as indicated by the markings in the battery compartment.

Then slide the cover back into its original position. Always use 2 new batteries (type AA, LR6) or equivalent rechargeable batteries.

Troubleshooting

No dial tone when the telephone handset is off the hook/green LED not lit:

- Check all plugs for correct connection:
Handset cord on telephone amplifier/receiver
Connection cord on telephone/telephone connection socket
- If necessary disconnect and reconnect all plugs (faulty contact).

Telephone connection operates but telephone amplifier does not function:

- Open the battery compartment and check if the batteries are inserted correctly.
- If necessary insert new batteries or fully charged rechargeable batteries

Low output volume:

- The batteries are depleted and need to be exchanged.

Adjustment of the PL-51

Cable layouts

As some telephones have different handset cable layouts, the *PL-51* may not function with such telephones on default. However, the *PL-51* can be modified to match different cable layouts.

To do this, the code board inside the battery compartment has to be replugged.

There are 4 possible positions for the code board: A, B, C and D.

Position A:

works with most telephones (default of the *PL-51*).

Position B, C and D:

further possible contact layouts.

Modifying the cable layout:

1. Unplug the telephone from the main telephone line socket.
2. Open the cover of the battery compartment.
3. Remove the two batteries.
4. Carefully pull the code board out of the slot and turn it.
5. Insert the code board back into the slot.
6. Insert the batteries (mind the polarity).
7. Close the battery compartment and plug the telephone back into the main telephone line socket.

Should your telephone amplifier still not work, repeat the above steps and try a different position of the code board.

Maintenance and care

The *PL-51* does not require any maintenance. If the unit does become dirty, simply wipe it clean with a soft, damp cloth. Never use spirits, thinners or other organic solvents. Do not set up the *PL-51* where it will be exposed to full sunlight for long periods. In addition, it must be protected against excessive heat, moisture and severe mechanical shocks.

Note: This product is not protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near

the product. Please make sure that the batteries are not exposed to excessive heat from such as sunlight, fire or anything similar.

Warranty

The *PL-51* is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge.

It is recommended that you send in the product in its original packaging, so keep the packaging for the duration of the warranty period.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit).

Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/ till receipt.

Always specify the product number in any event.

Disposal of used electric and electronic units (applicable in the countries of the European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collecting point for the recycling of electric and electronic units.

You protect the environment and health of your fellow men by the correct disposal of these products. Environment and health are endangered by a faulty disposal.

Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

Specifications

Dimensions:	40 x 70 x 105 mm (HxWxD)
Colour:	silver
Weight:	122 g incl. batteries
Amplification:	max. 40 dB \pm 20 %
Power supply:	2 x 1,5 V mignon batteries (Type LR6 or AA)

This product complies with the standards of the European Union.

CE compliance declarations are available on the Internet under www.humanteknik.com/service.

Technical specifications subject to change without prior notice.

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Sarabec Ltd