HUMANTECHNIK

*flashtel*  
eco

**D**

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Spezialtelefon

Seite 2

**GB**

**Operating Instructions**

Special telephone

Page 16

**F**

**Mode d'emploi**

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Page 30

**NL**

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Speciale telefoon

Pagina 44

**I**

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Telefono speciale

Pagina 58

**E**

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Página 72

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## Safety instructions

To enable you to get your telephone working quickly and to use it safely, it is essential that you read the following safety instructions and the section on "Getting started".

- Place the telephone on a non-slip surface.
- Lay the connecting cable safely in order to avoid the risk of accidents.
- Never expose the telephone to any of the following influencing factors:
  - sources of heat
  - direct sunlight
  - moisture
  - other electrical appliances
- Protect your telephone from damp, dust, aggressive liquids and steam.
- Only insert the connecting cable to the designated sockets/jacks.
- Only connect approved accessories to it.
- Never
  - open the telephone casing yourself!
  - carry the telephone by the connecting cables!

<b>Safety instructions</b>	page 16
<b>Getting started</b>	
Pack content	18
Telephone connection	18
<b>Controls</b>	
Controls on the front	19
Controls on the back / underside	20
Buttons and what they mean	21
<b>Using the telephone</b>	
Making calls	22
Answering calls	22
Using the telephone speaker during a call	22
Last number redial	22
Receiver volume / treble / bass control settings	23
HOLD	23
Inserting a dialing pause (PABX)	23
Inquiry button (HOOK/FLASH)	23
<b>Speed-dial buttons</b>	
Assigning numbers to the speed-dial buttons	24
Dialing with speed-dial buttons	24
<b>Settings</b>	
Switching the flashing light on/off	25
Setting the call signal volume	25
Tone/pulse dialing	25
<b>Additional ports</b>	
Headset port	25
Headphone port	25
<b>If your telephone doesn't work</b>	
General information	26
The telephone doesn't work	26
<b>Maintenance and care</b>	26
<b>Guarantee</b>	26
<b>CE-symbol</b>	27
<b>Environmental note / Waste disposal</b>	28
<b>Technical data</b>	28

## Getting started

### Pack content

The telephone comes with the following parts:

- Telephone base unit
- Telephone receiver
- Receiver cord
- Telephone cable
- Guarantee card
- Operating instructions
- Speed-dial inlay card
- Mains adapter

### Telephone connection

1. Insert the receiver cord (spiral-shaped cable) into the jack marked with a receiver symbol on the left side of the base unit and into the receiver.
2. Insert the smaller plug for the telephone cable into the "LINE" jack (back of the base unit). Insert the larger plug into the socket for your telephone line.
3. Insert the plug-in mains adapter cable into the "AC 9 V" jack. Plug the mains adapter into a 230 V mains socket.


### Warning!

Don't place the telephone close to televisions, VDUs, fans, etc. so as to minimize the risk of interference. Don't place the telephone in direct sunlight or close to a source of heat.


Only use the original telephone cable or a comparable cable with an international configuration.

# Controls


## Controls on the front

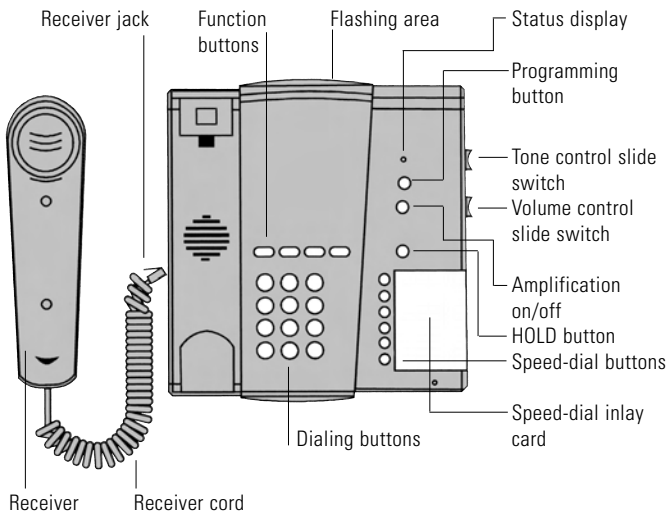## Controls on the back
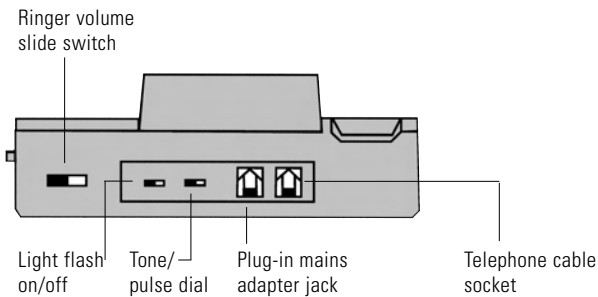
## Controls on the underside
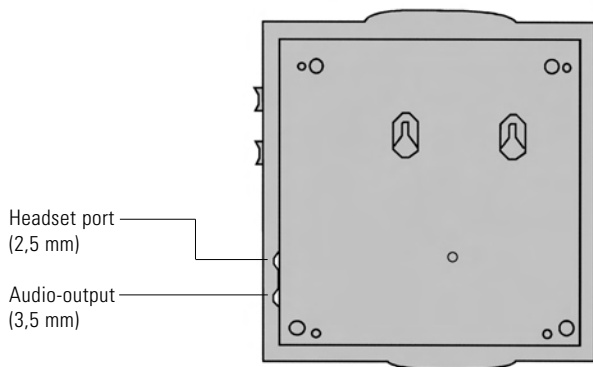
## Buttons and what they mean

Speaker button:

- "Hands-free talking": engage the line, hang up
- Enable/disable "conference" mode during a call

Inquiry button, e.g. for putting calls through if the telephone is connected to a telephone system

Dialing pause

Redial


Amplification on / off

Programming button

Hold button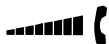

Volume control slide switch

Tone control slide switch

Speed-dial buttons

## Using the telephone

### Making calls

1. Lift up the receiver or press the  button.
2. Dial the telephone number.
3. In order to end the call, replace the receiver or press the  button again.

### Answering calls

1. The telephone rings.
2. Lift up the receiver or press the  button.
3. In order to end the call, replace the receiver or press the  button again.

### Using the telephone speaker during a call

1. Press the  button to activate the telephone speaker during a call. The receiver speaker remains activated.
2. You can now replace the receiver and continue the call over the telephone speaker and integrated microphone.
3. Press the  button to deactivate the telephone speaker again.

#### Note:

A call is ended if you press the  button while the receiver is on the cradle.

### Last number redial


1. Lift up the receiver.
2. If you press the  button, the last number called is redialed.

## Receiver volume/treble/bass control settings

The receiver volume can be adjusted during a call with the "Volume" slide switch on the right side of the telephone.

Additional amplification can be switched on by pressing the  button. When additional amplification is activated, the receiver volume can be raised to up to 30 dB. If amplification is switched on, the status display on the telephone lights up. The amplification also remains on when the receiver is placed back on its cradle. The additional amplification is deactivated by pressing the  button again.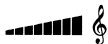

You can also increase treble or bass tones by an additional 10 dB using the tone control (slide switch on the right side of the base unit).

## HOLD

This function enables you to interrupt a conversation temporarily without ending the call. Press the  button to interrupt the conversation temporarily.

## Inserting a dialing pause (PABX)

Press the  button to interrupt dialing for 3.6 seconds.

## Inquiry button (Hook/Flash)

This function is required in order to activate certain T-Net features. If you press the  button, the telephone line is interrupted for a certain length of time (300 ms – fixed setting).

## Speed-dial buttons

### Assigning numbers to the speed-dial buttons

You can assign any telephone numbers to the 6 speed-dial buttons.

1. Lift up the receiver or press the  button.
2. Press the  button.
3. Enter the "telephone number" you require.
4. Press the  button again.
5. Assign a speed-dial button to the telephone number which you have entered by pressing one of the  ...  buttons.
6. Then replace the receiver or press the  button.

### Dialing with speed-dial buttons

Lift up the receiver or press the  button.

If you press any of the  ...  buttons, the telephone number which is stored there is dialed.

Note: If the telephone is disconnected from the mains/telephone network for more than 60 minutes, the stored telephone numbers will be deleted.

## Settings

### Switching the flashing light on/off

The telephone's visual call display (flashing light) can be switched on or off by means of a slide switch on the back of the telephone.

### Setting the call signal volume

The telephone's ringer volume can be set by means of a slide switch on the back of the telephone.

### Tone/pulse dialing

You can select the dialing process by means of a slide switch on the back of the telephone.

Note: Most telephone lines operate with tone dialing.

## Additional ports

### Headset port

Standard headset port with a 2.5 mm jack (for connecting a PL-100/ PL-200, for example).

### Headphone port

Output for connecting headphones, an induction loop or such like.

## **If your telephone doesn't work...**

### **General information**

Make sure that the telephone cable isn't damaged and is correctly inserted. Disconnect any additional equipment, extension cables and other telephones. If the telephone works then, this indicates that the fault is caused by another piece of equipment.

Try to connect your telephone to a neighbor's line, for example. If it works there, this indicates that there may be a fault on your telephone line. Call the service provider to report the fault.

### **The telephone doesn't work**

- Make sure that the mains adapter is correctly inserted into the telephone and into the mains socket.
- Make sure that the telephone cable is correctly inserted into the telephone and into the telephone socket.
- Connect another telephone to your telephone line. If it works, it indicates that your telephone is probably faulty.
- Only use the original cable or a telephone cable with an international configuration.

## **Maintenance and care**

The telephone is maintenance-free. If the telephone becomes dirty, wipe it occasionally with a soft, damp cloth. Never use alcohol, thinner or other organic solvents.

The telephone should never be exposed to sunlight for a long time and should also be protected from intense heat, moisture or strong mechanical vibrations.

## Important Notice

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases, or anything with an open flame, such as a lit candle, on or near the product.

## Guarantee

The telephone works very reliably. However, if faults occur despite being installed and operated properly, contact your specialist dealer or contact the manufacturer directly.

The guarantee covers free repair and return, provided it is returned in its original packaging. Therefore, do not throw this away.

The guarantee will become void in the case of damage resulting from improper handling or repairs attempted by unauthorized persons (damaging the telephone's seal).

Repairs will only be carried out under guarantee upon presentation of the completed guarantee card from the dealer or the original guarantee certificate in the case of telephones bought directly from us. In any event, the serial number of the telephone must also be indicated.

**CE symbol**

This telephone conforms to the requirements of EU Directive 1999/5/EC on radio equipment and telecommunications terminal equipment and the mutual recognition of their conformity.

Conformity with the afore-mentioned directive is confirmed by the CE symbol on the telephone.

## **Environmental note/Waste disposal**

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices).

The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment.

By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people.

The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

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## **Humantechnik GHL AG**