flashtel
comfort II **LB f**

D

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Seite 2

Spezialtelefon

GB

Operation Instructions

Page 30

Special telephone

F

Mode d'emploi

Page 58

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NL

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Pagina 86

Speciale telefoon

I

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Pagina 114

Telefono speciale

E

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Página 142

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Safety instructions

To enable you to get your telephone working quickly and to use it safely, it is essential that you read the following safety instructions and the section on „Getting started“.

- **Attention:**

Your telephone can be set to a high volume. To protect your hearing or the hearing of people with normal hearing, particularly of children, from any damage caused by excessively loud volume, please observe the following information:

1. Set the volume to low before you place the telephone receiver on your ear.
 2. While using the device, set the volume to a level that is comfortable for you.
- Place the telephone on a non-slip surface.
 - Lay the connecting cable safely in order to avoid the risk of accidents.
 - Never expose the telephone to any of the following influencing factors:
 - sources of heat
 - direct sunlight
 - moisture
 - Protect your telephone from damp, dust, aggressive liquids and steam.
 - Only insert the connecting cable to the designated sockets/jacks.
 - Only connect approved accessories to it.
 - Never
 - open the telephone casing yourself!
 - carry the telephone by the connecting cables!

Contents	page
Getting started	
Pack content	33
Telephone connection	33
Controls	
Controls on the front	34
Controls on the back / underside	35
Buttons and what they mean	36
Using the telephone	
Making calls	37
Answering calls	37
Using the telephone speaker during a call	37
Last number redial	37
Redial for the last 5 telephone numbers dialed	38
Deleting the last telephone number dialed	38
Deleting all telephone numbers dialed	38
Receiver volume / treble / bass control settings	39
Switch microphone amplifier on/off	39
HOLD	39
Inserting a dialing pause (PABX)	39
Inquiry button (HOOK/FLASH)	39
One-touch dial keys	
Allocating telephone numbers to one-touch dial keys	40
Using the one-touch dial keys	40
Deleting one of the one-touch dial keys	40
Deleting all one-touch dial keys	40
Telephone book	
Using the telephone book	41
Storing Name / Number	41
Placing a call with the telephone book	42
Deleting an entry in the telephone book	42
Deleting all the entries in the telephone book	42
Calling number display	
How it works	43
Reading and calling entries	43
Deleting an entry	43
Deleting all entries	43
Settings - Call signaling	
Switching the flash on/off	44
Switching the call signal on/off	44
Setting call signal volume	44
Call signal tone	44

Settings - Tone/Pulse dialing - Hook/Flash - PABX - PIN	
Tone / Pulse dialing	45
Hook / Flash	45
Pause - PABX	46
PIN (personal identification code)	46
SOS mode - General	
General	47
SOS call procedure	47
SOS mode - Programming	
Storing an SOS recorded message	48
Listening to an SOS recorded message	48
Storing SOS phone numbers	49
Setting the number of cycles	50
Pause after an unsuccessful cycle	50
Duration of the active connection	51
Assigning SOS hand-operated transmitters	51
SOS hand-operated transmitter range	52
Enabling/disabling SOS mode	52
Interrupting SOS mode	52
Emergency mode during power failure	52
Additional ports	
Headset port	53
Headphone port	53
Resetting the telephone to the factory settings	53
Power failure operation	54
If your telephone doesn't work	
General information	54
No number appears when the phone rings	54
The telephone doesn't work	54
Appendix	
Maintenance and care	55
Warranty	55
Environmental note/ Waste disposal	55
Technical data	56
CE-symbol	56

Getting started

Pack content

Please check if all following components are included:

- Telephone base unit
- Telephone receiver
- Receiver cord
- Telephone cable
- 9V plug-in PSU
- SOS hand-operated transmitter and pendant cord
- Velcro-fastening armband for the radio transmitter
- Speed-dial inlay card
- Operating instructions
- Guarantee card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Telephone connection


1. Insert the receiver cord (spiral-shaped cable) into the jack marked with a receiver symbol on the left side of the base unit and into the receiver.
2. Insert the smaller plug for the telephone cable into the "LINE" jack (back of the base unit). Insert the larger plug into the socket for your telephone line.
3. Insert the plug-in mains adapter cable into the "DC 9 V" jack. Plug the mains adapter into a 230 V mains socket.

Warning!


Don't place the telephone close to televisions, VDUs, fans, etc. so as to minimize the risk of interference. Don't place the telephone in direct sunlight or close to a source of heat. Only use the original telephone cable or a comparable cable with an international configuration.

Controls


Controls on the front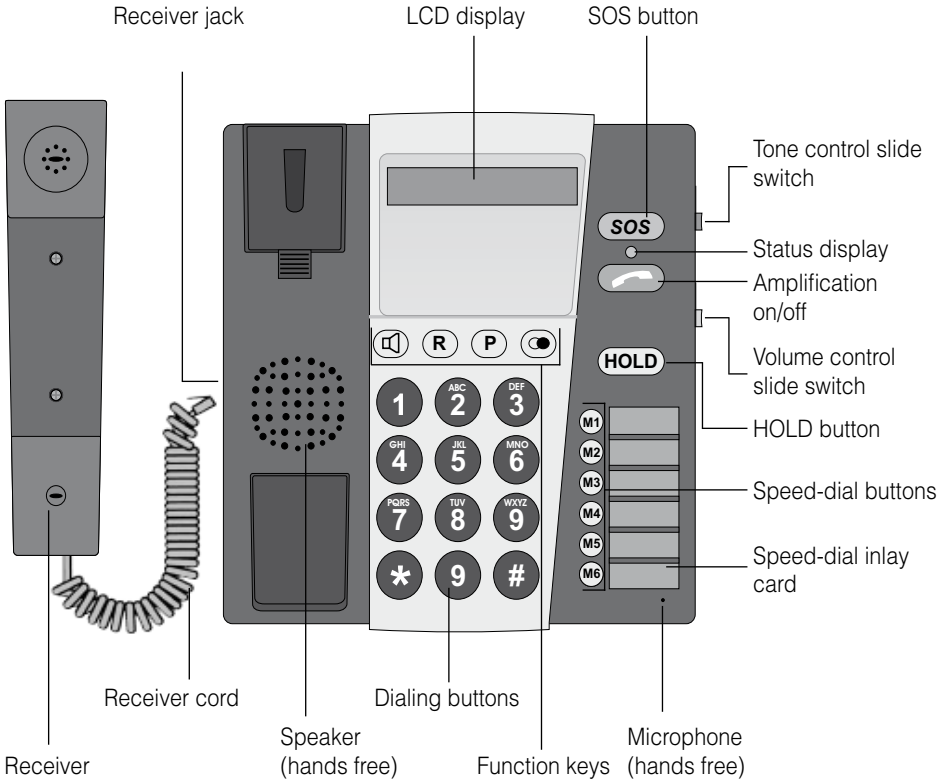

Controls on the back

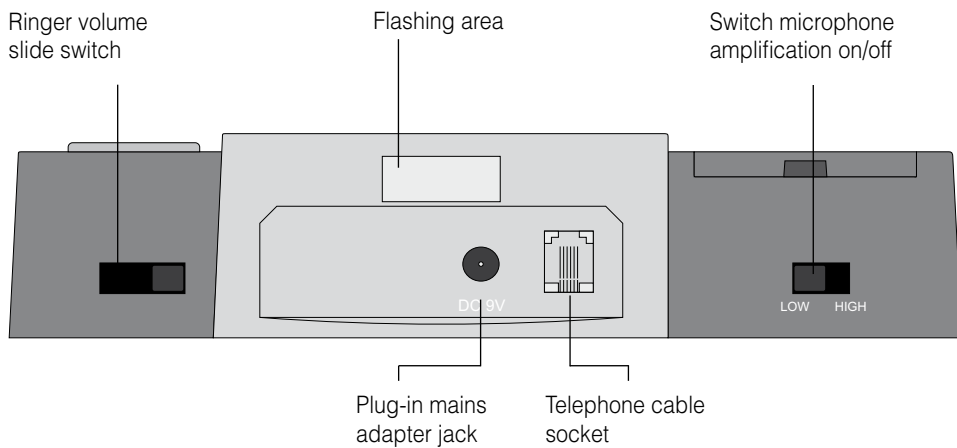Controls on the underside

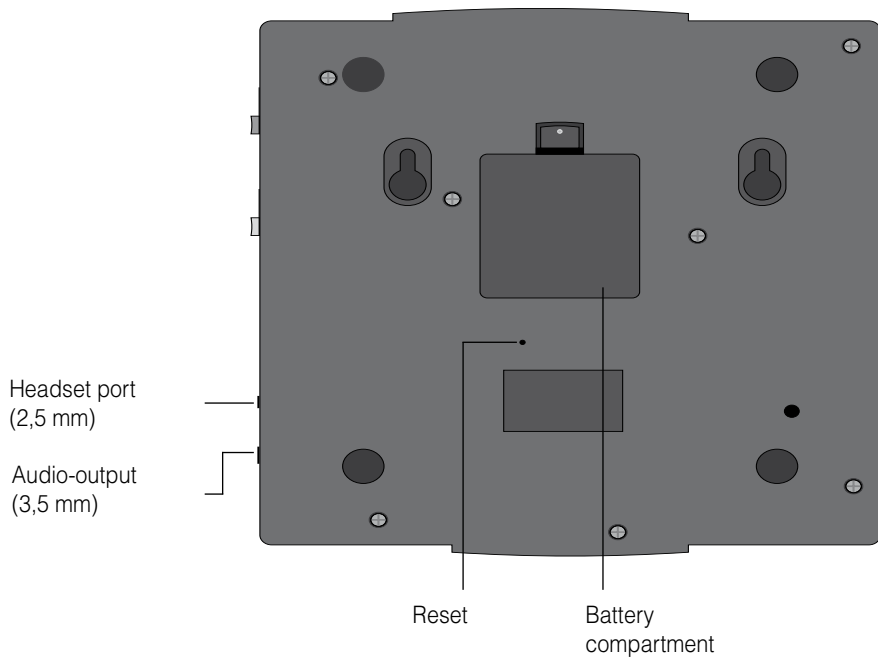Buttons and what they mean

Speaker button:

- „Hands-free talking“: engage the line, hang up
- Enable/disable „conference“ mode during a call

Inquiry button, e.g. for putting calls through if the telephone is connected to a telephone system

Programming key: Initiating operating procedure and storing


Redial

SOS button

Amplification on / off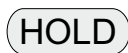


Hold button

Volume control slide switch

Tone control slide switch

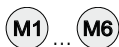Speed-dial buttons

Initiate programming for settings without PIN

Initiate programming for settings with PIN

Using the telephone

Making calls

1. Lift up the receiver or press the  button.
2. Dial the telephone number.
3. In order to end the call, replace the receiver or press the  button again.

Answering calls

1. The telephone rings.
2. Lift up the receiver or press the  button.
3. In order to end the call, replace the receiver or press the  button again.

Using the telephone speaker during a call

1. Press the  button to activate the telephone speaker during a call. The receiver speaker remains activated.
2. You can now replace the receiver and continue the call over the telephone speaker and integrated microphone.
3. Press the  button to deactivate the telephone speaker again.

Note:

A call is ended if you press the  button while the receiver is on the cradle.

Last number redial

1. Lift up the receiver.
2. If you press the  button, the last number called is redialed.

Redial for the last 5 telephone numbers dialed

The last 5 telephone numbers dialed will be stored in the telephone automatically.

1. Press .
2. Use  and  to page between the entries.
3. Once the number you are looking for appears in the display, lift the receiver. The number you have selected will be dialed automatically.

Note:

The receiver must be on the cradle while you are selecting the telephone number you want to redial.

Deleting the last telephone number dialed

1. Press .
2. Use  and  to page between the entries.
3. Press  to delete the entry currently selected. "OK ?" will appear in the telephone display. Press  to confirm your selection.

Deleting all the telephone numbers dialed

1. Press .
2. Press and hold  for 2 seconds. "OK ?" will appear in the telephone display. Press  to confirm your selection.

Receiver volume/treble/bass control settings

The receiver volume can be adjusted during a call with the “Volume” slide switch on the right side of the telephone.

Additional amplification can be switched on by pressing the  button. When additional amplification is activated, the receiver volume can be raised to up to 30 dB. If the amplification is switched on, the status display will light up along with the symbol  in the telephone display.

The amplification also remains on when the receiver is placed back on its cradle. The additional amplification is deactivated by pressing the  button again.

You can also increase treble or bass tones by an additional 10 dB using the tone control (slide switch on the right side of the base unit).

Switch microphone amplifier on/off

The «flashtel comfort II LB f» is equipped with adjustable microphone amplification to enable you to better understand the person you are speaking to when the sound is quiet. To switch on the microphone amplification, simply slide the switch on the rear panel of the telephone to “HIGH”.

HOLD

This function enables you to interrupt a conversation temporarily without ending the call. Press the  button to interrupt the conversation temporarily.

Inserting a dialing pause (PABX)

Press the  button to interrupt dialing for 1-6 seconds.

Inquiry button (Hook/Flash)

This function is required in order to activate certain T-Net features. If you press the  button, the telephone line is interrupted for a certain length of time. Fixed setting standard 260 ms.

One-touch dial keys

Allocating telephone numbers to one-touch dial keys

The 6 speed-dial keys can be programmed with any telephone number.

1. Press **(P)** and then **(#)**.
2. Enter the code “**01**” and the desired “telephone number”. The telephone number will be allocated to the **M1** key.
3. Enter the code “**02**” and the desired “telephone number”. The telephone number will be allocated to the **M2** key.
4. Enter the code “**03**” and the desired telephone number. The telephone number will be allocated to the **M3** key.
5. Enter the code “**04**” and the desired “telephone number”. The telephone number will be allocated to the **M4** key.
6. Enter the code “**05**” and the desired “telephone number”. The telephone number will be allocated to the **M5** key.
7. Enter the code “**06**” and the desired “telephone number”. The telephone number will be allocated to the **M6** key.
8. Press **(P)** to store the changes.
Enter another code or press **(P)** to quit the settings menu.

Using the one-touch dial keys

Take the receiver from the cradle or press the key **(📞)**.

If you press one of the keys marked **(M1)** **(M6)** the telephone number stored for that key will be dialed automatically.

Deleting one of the one-touch dial keys

1. If you press any of the **(M1)** **(M6)** buttons, the telephone number which is stored there is dialed.
2. Press **(R)** to delete the entry selected. “**OK ?**” will appear in the telephone display. Press **(P)** to confirm your selection.

Deleting all one-touch dial keys

1. If you press any of the **(M1)** **(M6)** buttons, the telephone number which is stored there is dialed.
2. Press the key marked **(R)** for 2 seconds to delete all the speed-dial entries. “**OK ?**” will appear in the telephone display. Press the key marked **(P)** to confirm your selection.

Telephone book

Using the telephone book

Names and numbers can be stored in the telephone book. Once a telephone number is stored, you can call that number by pressing a minimum number of keys. Up to 30 telephone numbers can be stored in the telephone book. Each telephone number can contain up to 20 digits.

Storing name/number

To make new entries in the telephone book, proceed as follows:

1. Press **(P)** and then **(#)**.
2. Enter the code **"80"** for a new entry.
3. Enter the desired telephone number on the keypad. Press **(R)** to delete the last digit entered.
4. Press **(P)** to store the telephone number and to begin to enter the name.
5. Press the corresponding key as frequently as necessary to select the desired letter (see table below). Press **(R)** to delete the last letter entered.
6. Press **(P)** to complete the programming process.

Key	Letter / character				
1	1				
2	A	B	C	2	
3	D	E	F	3	
4	G	H	I	4	
5	J	K	L	5	
6	M	N	O	6	
7	P	Q	R	S	7
8	T	U	V	8	
9	W	X	Y	Z	9
0	-	0			

Placing a call with the telephone book

1. Press **(P)** and then **(#)**.
2. Enter the code “**81**” to access the telephone book.
3. Press **(*)** and **(#)** to page between the entries.
4. Lift the receiver or press the key **(📞)** to dial the number you have selected.

Deleting an entry in the telephone book

1. Page to the desired entry as described in “Placing a call with the telephone book”.
2. Press **(R)** to delete the entry selected. “**OK ?**” will appear in the telephone display. Press **(P)** to confirm your selection.

Deleting all the entries in the telephone book

1. Access the telephone book as described in “Placing a call using the telephone book”.
2. Press and hold **(R)** for 2 seconds. “**OK ?**” will appear in the telephone display. Press **(P)** to confirm your selection.

Calling number display

How it works

The number of the person calling you will appear in the telephone display. The telephone supports both of the current systems (DTMF/ FSK). Your telephone will store the last 5 calling numbers automatically. If the memory is full, the oldest entry will be deleted.

Important!

Calling number display will only work if this service is supported by your telecom supplier. Please inquire with your telephone company.

Reading and calling entries

The last 5 telephone numbers recognized will be automatically stored in the telephone.

1. Press **R**.
2. Use ***** and **#** to page between the entries. (Only the first 10 digits of the telephone number will appear in the display. Press **☎** to see the rest of the telephone number. Press the key **☎** again to see the date and time the call was received).
3. Once the desired telephone number appears in the display, lift the receiver or press the **☎**. The number you have selected will be dialed.

Note:

The receiver must be resting on the cradle when you select the telephone number.

Deleting an entry

1. Page to the desired entry as described in "Reading and calling entries".
2. Press **R** to delete the entry selected. "OK ?" will appear in the telephone display. Press **P** to confirm your selection.

Deleting all entries

1. Access the telephone book as described in "Reading and calling entries".
2. Press and hold **R** for 2 seconds. "OK ?" will appear in the telephone display. Press **P** to confirm your selection.

Settings – Call signaling

Switching the flash on/off

You can also have incoming calls signaled by a flashing light. The flashing light appears on the back of the telephone.

1. Press **(P)** and then **(#)**.
2. Then enter one of the two following codes:
 - a. Code **"40"** to switch the flash off.
 - b. Code **"41"** to switch the flash on;
the flash symbol will appear in the telephone display.
3. Press **P** to store the settings.
4. Enter another code or press **P** to quit the settings menu.

Note:

The flash will not work during a power failure!

Switching the call signal on/off

1. Press **(P)** and then **(#)**.
2. Enter one of the following two codes:
 - a. Code **"20"** switches the call signal off.
 - b. Code **"21"** switches the call signal on.
3. Press **(P)** to store the settings.
4. Enter another code or press **(P)** to quit the settings menu.

Setting the volume of the call signal

The volume of the call signal can be adjusted by moving the sliding switch on the back of the telephone.

Call signal tone

You can select from between 7 different acoustic call signals.

1. Press **(P)** and then **(#)**.
2. Enter code **"23"**. The current signal setting will appear in the display.
3. Press number keys 1-8 to alter the call signal.
4. Press **(P)** to store the settings.
5. Enter another code or press **(P)** to quit the settings menu.

Settings - Tone/pulse dialing - Hook/Flash - PABX - PIN

Tone/pulse dialing

1. Press **(P)** and then **(#)**.
2. Enter one of the following three codes:
 - a. Code "**10**" for tone dialing
 - b. Code "**11**" for pulse dialing (33% M/B)
 - c. Code "**12**" for pulse dialing (40% M/B)
3. Press **(P)** to store the settings.
4. Enter another code or press **(P)** to quit the settings menu.

Note:

Nowadays, most telephone connections use tone dialing.

Hook/Flash

You can set the flash key to different flash times. The flash setting usually does not have to be altered when the telephone is connected to a larger telephone system (see the operating manual for the telephone system).

1. Press **(P)** and then **(*)**.
2. Enter the four-digit unit PIN "**XXXX**" (the default PIN code for the telephone is "**0000**").
3. You can now enter one of the following codes:
 - a. Code "**700**" sets the flash time to "**90 ms**",
 - b. Code "**701**" sets the flash time to "**110 ms**",
 - c. Code "**702**" sets the flash time to "**130 ms**",
 - d. Code "**703**" sets the flash time to "**260 ms**" or
 - e. Code "**704**" sets the flash time to "**600 ms**".
4. Press **(P)** to store the settings.
5. Enter another code or press **(P)** to quit the settings menu.

Pause - PABX

You can alter the length of the dialing pause (1-6 seconds). The pause usually does not have to be altered when the telephone is connected to a larger telephone system (see the operating manual for the telephone system).

1. Press **(P)** and then **(*)**.
2. Enter the four-digit unit PIN "**XXXX**". (The default PIN code for the telephone is "**0000**").
3. Enter code „**720**“.
4. Enter the dialing pause with the number keys 1-6.
5. Press **(P)** to store the settings.
6. Enter another code or press **(P)** to quit the settings menu

PIN (Personal Identification Code)

Some of your telephone's functions are protected by a PIN. The default PIN for your telephone is "**0000**".

1. Press **(P)** and then **(*)**.
2. Enter the four-digit unit PIN "**XXXX**". (The default PIN code for the telephone is "**0000**").
3. Enter code "**600**".
4. Now enter the new four-digit PIN "**XXXX**".
5. To confirm the entry of your new PIN, enter it once again.
6. The PIN will be stored automatically once you have entered the last digit.
7. Enter another code or press **(P)** to quit the settings menu.

SOS mode - General

General

SOS mode must be enabled via programming. You can store up to 8 phone numbers which are dialed in consecutive order if an SOS call is triggered on the telephone or via the SOS hand-operated transmitter provided.

Other SOS hand-operated transmitters are optionally available as special accessories.

In SOS mode, the SOS call has priority over all the other telephone functions. This means that incoming calls are blocked if an SOS call is transmitted at the same time.

You can record an individual SOS recorded message. The recipient of the SOS call hears the recorded message and knows what to do.

SOS call procedure

Please ensure that the recipient of the SOS call has their telephone set to DTMF dialing mode. This is a requirement for establishing a speech connection with the person who is making the SOS call.

SOS caller

Press the SOS button on the telephone or transmit the SOS call using the SOS hand-operated transmitter.

The SOS telephone is automatically switched to hands-free mode.

Press the  button to end the call.

SOS call recipient

The telephone rings. Pick up the receiver. Listen to the SOS recorded message. Press 5 (acknowledge) to stop the SOS recorded message.

Talk to the SOS caller.

SOS mode – Programming

When you enable the SOS mode setting, you should program the following settings:

1. Store an SOS recorded message
2. Store the SOS phone numbers
3. Set the number of cycles
4. Set the length of pause after an unsuccessful cycle
5. Set the duration of the active connection
6. Assign SOS hand-operated transmitters
7. Enable SOS mode

Storing an SOS recorded message

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN „XXXX“ (the telephone's default PIN code is „0000“).
3. If you enter the code „354“, „Start“ appears on the display.
4. Keep the button **(C)** pressed until „ERASING“ disappears in the display. Now you can record the SOS recorded message (max. 20 seconds; time is shown).
Keep the button **(C)** pressed during your record.
5. Release the button **(C)** again to stop recording.
Re-enter a code or press the **(P)** button to exit the Settings menu.

Note:

When an SOS call is received, the SOS recorded message is automatically played. On the one hand, the SOS recorded message should be kept short (max. 20 seconds), although it should contain all the information that the recipient requires in order to take immediate steps to provide aid quickly.

Listening to an SOS recorded message

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN „XXXX“ (the telephone's default PIN code is „0000“).
3. If you enter the code „355“, „Start“ appears on the display.
4. Press the **(C)** button. „Play back“ appears on the display, and the recording is played.
Re-enter a code or press the **(P)** button to exit the Settings menu.

Storing the SOS phone numbers

Up to 8 different phone numbers can be stored in the SOS call register. If an SOS call is triggered, the SOS phone numbers are called in consecutive order. Each SOS phone number can be up to 20 digits long.

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN "**XXXX**" (the telephone's default PIN code is "**0000**").
3. **Memory slot 1:** enter the code "**311**", followed by the requested SOS phone number, or
4. **Memory slot 2:** enter the code "**312**", followed by the requested SOS phone number, or
5. **Memory slot 3:** enter the code "**313**", followed by the requested SOS phone number, or
6. **Memory slot 4:** enter the code "**314**", followed by the requested SOS phone number, or
7. **Memory slot 5:** enter the code "**315**", followed by the requested SOS phone number, or
8. **Memory slot 6:** enter the code "**316**", followed by the requested SOS phone number, or
9. **Memory slot 7:** enter the code "**317**", followed by the requested SOS phone number, or
10. **Memory slot 8:** enter the code "**318**", followed by the requested SOS phone number.
11. Press the **(P)** button to save the changes.
Re-enter a code or press the **(P)** button to exit the Settings menu.

Note:

There must be at least one SOS phone number stored for an SOS call to be transmitted.

Setting the number of cycles

A cycle is defined as dialing all the programmed SOS phone numbers in one consecutive sequence. By setting the number of cycles you will define how frequently the phone numbers are to be dialed.

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN “XXXX” (the telephone’s default PIN code is “0000”).
3. Enter the code “350”.
4. Enter the number of cycles using the number keys 1-8.
If you press 9, the telephone will keep transmitting the SOS call until it is answered.
5. Press the **(P)** button to save the changes.
6. Re-enter a code or press the **(P)** button to exit the Settings menu.

Note:

The telephone’s default is 3 cycles.

Pause after an unsuccessful cycle

You can specify the pause (1-9 minutes) between the end of an unsuccessful cycle and the start of a new cycle.

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN “XXXX” (the telephone’s default PIN code is “0000”).
3. Enter the code “351”.
4. Enter the duration of the pause using the number keys 1-9.
5. Press the **(P)** button to save the changes.
6. Re-enter a code or press the **(P)** button to exit the Settings menu.

Note:

The SOS LED flashes during the pause between cycles and is illuminated constantly during cycles.

Duration of the active connection

The “duration of the active connection” is the length of time during which an SOS phone number is dialed. If no-one answers the SOS call, the call ends after max. 90 seconds (the telephone’s default is 90 seconds). If the recipient answers the call after 60 seconds for example, he hears the SOS recorded message for a maximum of 30 seconds.

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN “XXXX” (the telephone’s default PIN code is “0000”).
3. Enter the code “352”.
4. Enter the duration of the active connection using the number keys 3-9 (max. 90 seconds).
5. Press the **(P)** button to save the changes.
6. Re-enter a code or press the **(P)** button to exit the Settings menu.

Assigning SOS hand-operated transmitters

The SOS hand-operated transmitter must be assigned to the telephone in order to work with it. Procedure for assigning the SOS hand-operated transmitter:

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN “XXXX” (the telephone’s default PIN code is “0000”).
3. Enter one of the following codes:
 - a. Code “381”; R1 appears in the display to indicate the first SOS transmitter.
 - b. Code “382”; R2 appears in the display to indicate the second SOS transmitter.
 - c. Code “383” ... code “388”; R3...R8 appear in the display to indicate the third to eighth SOS transmitters, depending on which code is selected.
4. Press the button on the SOS hand-operated transmitter twice. Once it is successfully assigned, R1...R8 disappear from the display.
5. Re-enter a code or press the **(P)** button to exit the Settings menu.

Note:

A total of up to eight SOS hand-operated transmitters can be assigned to the telephone.

SOS hand-operated transmitter range

We recommend testing the range of the SOS hand-operated transmitter because this may vary depending on the nature of the building. If you know the precise range, this will safeguard against malfunctions. It has a range of up to 30 meters inside buildings and up to 80 meters outdoors.

Enabling/disabling SOS mode

1. Press the **(P)** button and then the **(*)** button.
2. Enter the four-digit device PIN “XXXX” (the telephone’s default PIN code is “0000”).
3. Now enter one of the following two codes:
 - a. Code “300” disables SOS mode.
 - b. Code “301” enables SOS mode.
5. Press the **(P)** button to save the changes.
6. Re-enter a code or press the **(P)** button to exit the Settings menu.

Note:

There must be at least one SOS phone number stored for an SOS call to be transmitted.

Interrupting SOS mode

Press the **(P)** button and then the **(#)** button to interrupt an SOS call which is triggered accidentally.

Emergency mode during power failure

If five AAA batteries are inserted into the telephone’s battery compartment, the telephone can also be contacted during a power failure.

Ensure that the plug-in PSU is connected to the mains. Otherwise, the batteries will become discharged very quickly.

Additional ports

Headset port

Standard headset port with a 2.5 mm jack (for connecting a PL-100/ PL-200, for example).

Headphone port

Output for connecting headphones, an induction loop or such like.

Resetting the telephone to the factory settings

You can press a button on the back of the telephone to reset all the telephone settings to the default settings. Press and hold this button for 2 seconds. This function is useful if you have forgotten the telephone's PIN.

Default settings:

Flashing light:	ON
One-touch dial keys:	None
Redial memory:	Empty
Calling number display memory (CLIP):	Empty
Telephone book:	Empty
Hook/Flash:	260 ms
Pause:	3 s
Ring tone:	1
Tone/pulse dialing:	Tone dialing
PIN code:	0000


Power failure operation

Should there be a power failure, you can still use your telephone. However, the flashing light will not work.

If you would like to use the display function during a power failure, you can put 5 batteries (micro or AAA) into the telephone.

Note:

During a power failure, the telephone's flashing light will not work even if you can put in batteries.

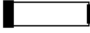The dead-battery symbol  will appear in the display if the batteries are dead or not inserted into the device.

If your telephone doesn't work...

General information

Make sure that the telephone cable isn't damaged and is correctly inserted. Disconnect any additional equipment, extension cables and other telephones. If the telephone works then, this indicates that the fault is caused by another piece of equipment. Try to connect your telephone to a neighbor's line, for example. If it works there, this indicates that there may be a fault on your telephone line. Call the service provider to report the fault.

No number appears when the phone rings

- Calling number display is a function that has to be supported by your service provider.
- If no calling number appears in the display, this means that no calling number information is available or that the caller has an unlisted number. This can also happen when a call is received from a telephone exchange.
- If the telephone is connected to an extension, the calling number display function will not usually work.

The telephone doesn't work

- Make sure that the mains adapter is correctly inserted into the telephone and into the mains socket.
- Make sure that the telephone cable is correctly inserted into the telephone and into the telephone socket.
- Connect another telephone to your telephone line. If it works, it indicates that your telephone is probably faulty.
- Only use the original cable or a telephone cable with an international configuration.

Appendix

Maintenance and care

The telephone is maintenance-free. If the telephone becomes dirty, wipe it occasionally with a soft, damp cloth. Never use alcohol, thinner or other organic solvents.

The telephone should never be exposed to sunlight for a long time and should also be protected from intense heat, moisture or strong mechanical vibrations.

Warranty

The telephone is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**

Environmental note/ Waste disposal

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and other European countries with a separate collection system for these devices).

The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment. By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

Technical data

Power supply:

Plug-in PSU:

Primary 100-240 V 50 Hz / 180 mA -

Secondary 9 V DC / 300 mA

Dimensions (LxBxH):

210 mm x 227 mm x 70 mm

Weight:

860 g

Hand-operated transmitters:

Carrier Frequency:

869,220 MHz +/- 5 kHz

Modulation:

FSK

Radiated Power:

≤ 10mW e.r.p.

Operation Voltage Range:

4,8~6V

Working current:

<20mA

Stand by current:

<10µA

Transmitting On Time:

<0,72s

Crystal Accuracy:

<10ppm

Encoding:

HS1527

CE symbol

This telephone satisfies the following EU directives:

- 2002/95/EG RoHS directive
- 2002/96/EG WEEE directive
- 1999/5/EG R&TTE directive
- 2004/108/EG EMC directive
- 2006/95/EG Low voltage directive

Compliance with the directives listed above is confirmed by the CE seal on the telephone. CE compliance declarations are available on the Internet at www.humanteknik.com.

Technical specifications subject to change without notice.

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