HUMANTECHNIK

flash**TEL**comfort**3 BT**

DE

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Seite 2

Spezialtelefon

EN

Operation Instructions

Page 28

Special telephone

FR

Mode d'emploi

Page 54

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NL

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Pagina 80

Speciale telefoon

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Safety instructions

To enable you to get your telephone working quickly and to use it safely, it is essential that you read the following safety instructions and the section on „Getting started“.

- **Attention:**

Your telephone can be set to a high volume. To protect your hearing or the hearing of people with normal hearing, particularly of children, from any damage caused by excessively loud volume, please observe the following information:

1. Set the volume to a low level before you place the telephone handset on your ear.
 2. While using the device, set the volume to a level that is comfortable for you.
- Place the telephone on a non-slip surface.
 - Lay the connecting cable safely in order to avoid the risk of accidents.
 - Never expose the telephone to any of the following influencing factors:
 - sources of heat
 - direct sunlight
 - moisture
 - Protect your telephone from damp, dust, aggressive liquids and steam.
 - Only insert the connecting cable to the designated sockets/jacks.
 - Only connect approved accessories to it.
 - Never
 - open the telephone casing yourself!
 - carry the telephone by the connecting cables!

Pacemaker Safety

The flashtel comfort 3 BT is equipped with a permanent magnet for the “Autodetect” function of hearing aids.

Attention:

1. Before using the flashtel comfort 3 BT with a pacemaker or other medical device, consult your physician or the manufacturer of your pacemaker or other medical device.
2. If you have a pacemaker or other medical device, make sure that you are using this product in accordance with safety guidelines established by your physician or the pacemaker manufacturer.

Contents	page
Getting started	
Pack content	31
Telephone connection	31
Installing the batteries	31
Controls	
Controls on the front	32
Controls on the left / back / right	33
Buttons and their functions	34
Using the telephone	
Making calls	35
Answering calls	35
Using the telephone speaker during a call	35
Setting the tone	35
Last number redial	36
Redial for the last 10 telephone numbers dialed	36
Store one of the last 10 telephone number dialed	36
Deleting the last telephone number dialed	36
Deleting all telephone numbers dialed	37
Handset volume	37
Switching the microphone off/on	37
HOLD	37
Inserting a dialing pause (PABX)	37
Inquiry button (Recall time)	37
Telephone book	
Using the telephone book	38
Storing name/number	38
Placing a call with the telephone book	39
Editing an entry in the telephone book	39
Deleting an entry in the telephone book	39
One-touch dial keys	
Allocating telephone numbers to one-touch dial keys	40
Using the one-touch dial keys	40
Deleting one of the one-touch dial keys	40
Calling number display	
How it works	41
Reading and calling entries	41
Storing an entry	41
Deleting an entry	42
Deleting all entries	42

Settings

Language - Date format - Time format - Date & time-

Ring volume - Ring melody

Setting the language	43
Setting the date format	43
Setting the time format	43
Setting the date & time	43
Setting the ring volume	44
Setting the ring melody	44

Key tone - Contrast - Recall time (Flash)

Setting the key tone	45
Setting the contrast	45
Setting the recall time (Flash)	45

Dynamic font - Dialing mode - Call waiting -

Auto On Hook - Auto Amp - Mic Amp

Setting the dynamic font	46
Setting the dialing mode	46
Setting the call waiting	46
Setting the Auto On Hook	47
Setting the Auto Amp function	47
Setting the Mic Amp function	47

Bluetooth

Pairing of a Bluetooth headset	48
Connecting of a Bluetooth headset	48
Disconnecting of a Bluetooth headset	49
Removing of a paired headset	49

Additional ports

Connection jack for vibrating pillow and/or modules	50
Headset port	50
Headphone port	50

Resetting the telephone to the factory settings

Power failure operation

Troubleshooting

General information	51
No number appears when the phone rings	51
The telephone doesn't work	51

Appendix

Maintenance and care	52
Warranty	52
Environmental note/ Waste disposal	52
Technical data	53
CE-symbol	53

Getting started

Pack content

Please check if all following components are included:

- Telephone base unit
- Telephone handset
- Handset cord
- Telephone cable
- 7.5V plug-in PSU
- Operating instructions
- Guarantee card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

Telephone connection

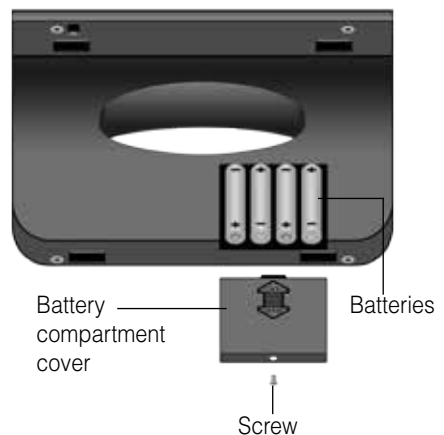
1. Insert the handset cord (spiral-shaped cable) into the jack marked with a receiver symbol on the left or right side of the base unit and into the handset.
2. Insert the smaller plug for the telephone cable into the "LINE" jack (back of the base unit). Insert the larger plug into the socket for your telephone line.
3. Insert the plug-in mains adapter cable into the "DC 7.5V" jack. Plug the mains adapter into a 230 V mains socket.

Warning!

Don't place the telephone close to televisions, VDUs, fans, etc. so as to minimize the risk of interference. Don't place the telephone in direct sunlight or close to a source of heat. Only use the original telephone cable or a comparable cable with an international configuration.

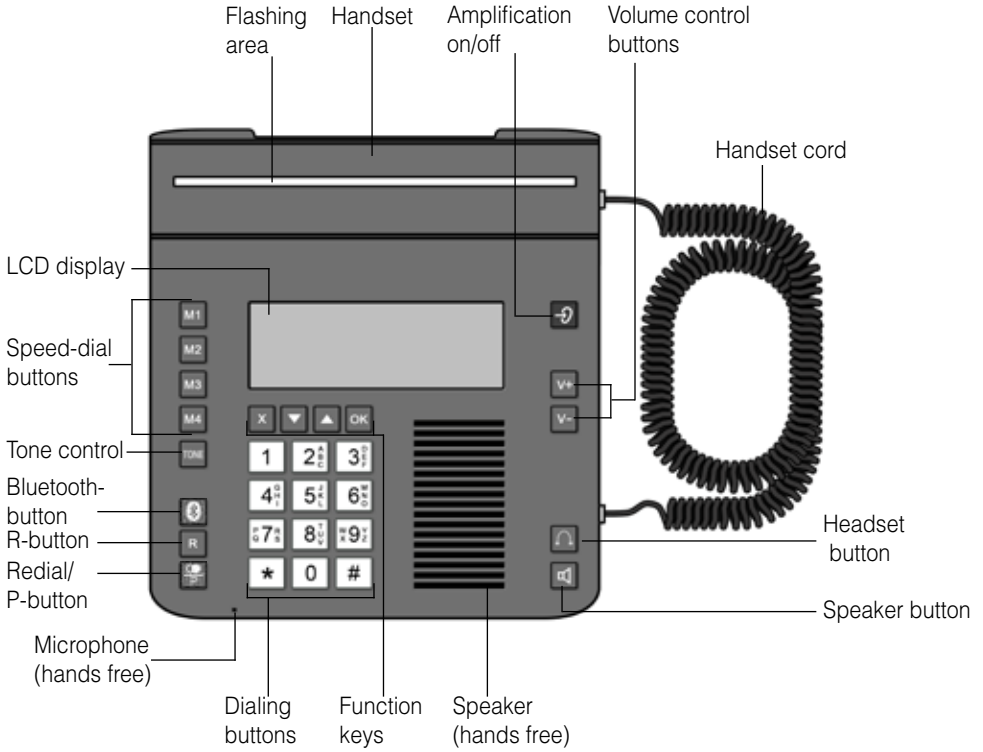
Installing the batteries

1. Remove the battery compartment cover by unscrewing the screw and sliding the cover off.
2. Insert 4 fresh AA size batteries while observing the +/- polarity marks in the battery compartment.
3. Replace the battery compartment cover and secure it with the screw.
4. Connect the telephone cable to the wall outlet for telephone.

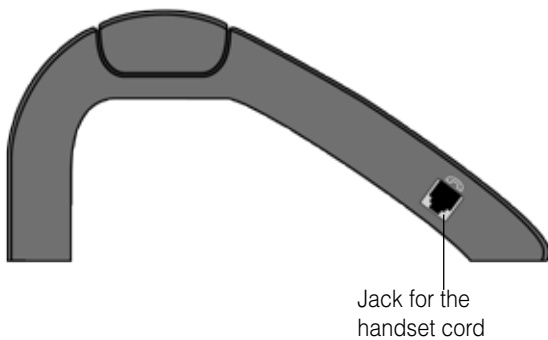
Controls

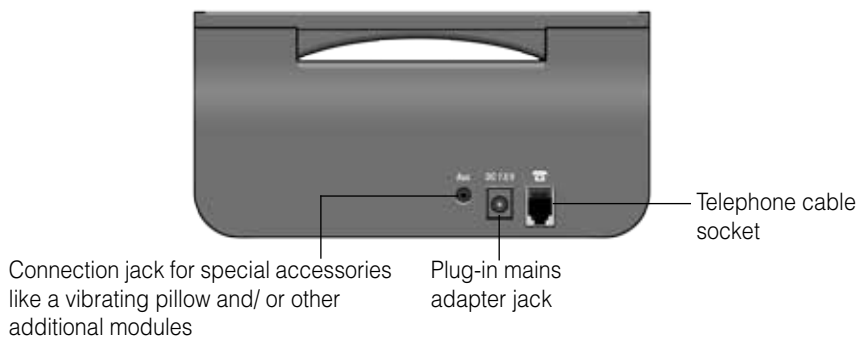
Controls on the front

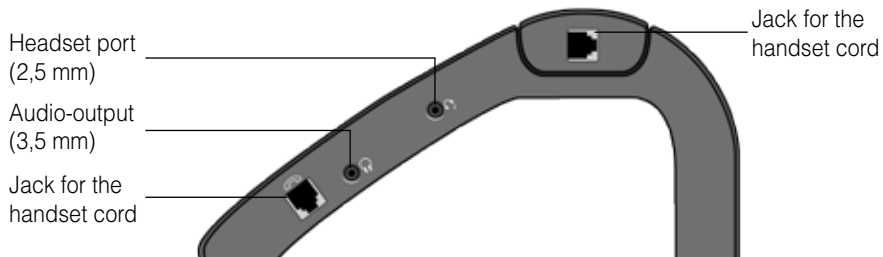
Controls on the left

Controls on the back

Controls on the right

Buttons and their functions

Speed-dial buttons

Tone control during a phone call

Bluetooth button:

- Making a phone call over a connected Bluetooth headset
- Engage the line, hang up

Inquiry button, e.g. for putting calls through if the telephone is connected to a telephone system

Redial / Pause key: Insert pause while dialing a number

Back / Delete last entry/ Phonebook/ Switching the microphone off/on

Arrow keys down/up: Scroll through menu

OK button: Confirm selection, Select menu, HOLD button

To enter the digits 0-9

To enter *

To enter #

Amplification on / off

Volume control button

Headset button:

- Making a phone call over a connected Headset
- Engage the line, hang up

Speaker button:

- „Hands-free talking“: engage the line, hang up
- Enable/disable „conference“ mode during a call

Using the telephone

Making calls

1. Lift up the handset or press the  button. If you want to make a phone call over a connected headset press .
2. Dial the telephone number.
3. In order to end the call, replace the handset or press either the  button or the  button.

Answering calls

1. The telephone rings.
2. Lift up the handset or press the  button. If you want to make a phone call over a connected headset press .
3. In order to end the call, replace the handset or press either the  button or the  button.

Using the telephone speaker during a call

1. Press the  button to activate the telephone speaker during a call.
2. You can now replace the handset and continue the call over the telephone speaker and integrated microphone.
3. To deactivate the telephone speaker again lift up the handset.

Note:

A call is ended if you press the  button while the handset is on the cradle.

Setting the tone

1. Press the  button to adjust the tone during a phone call.
2. The tone can be adjusted in four steps by pressing the  button.

Note:

The setting of the tone works only if the handset is lift up or with a connected headset. During a call over the telephone speaker the tone can not be adjusted.

Last number redial

1. Press the  button. The last dialed number will be shown on the display.
2. Lift up the handset or press either the  button or the  button if a headset is connected. The number is redialed.

Redial for the last 10 telephone numbers dialed

The last 10 telephone numbers dialed will be stored in the telephone automatically.

1. Press .
2. Use  or  to page between the entries.
3. Once the number you are looking for appears in the display, lift the handset or press either the  button or the  button if a headset is connected. The number you have selected will be dialed automatically.

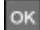
Note:

The handset must be on the cradle while you are selecting the telephone number you want to redial.

Store one of the last 10 telephone number dialed

1. Press .
2. Use  or  to page between the entries.
3. Once the number you are looking for appears in the display, press . Use  or  to page to “Store”.
4. Choose “Store” to store the chosen entry and press .
5. “Number:” and the phonenumber will be shown in the display. Press  to confirm.
6. Now “Name:” will be shown and you can enter a name for the telephone number.
7. After you have entered the name, press .
8. Now you can select a ringtone for your entry.
9. Press  to confirm and save your entry.

Deleting the last telephone number dialed

1. Press .
2. Use  or  to page between the entries.
3. Once the number you are looking for appears in the display, press . “Erase” will appear in the telephone display.
4. Press  to delete the chosen entry.

Deleting all the telephone numbers dialed

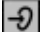
1. Press .
2. Press  and use ▼ or ▲ to page between “Erase”, “Erase all” and “Store” .
3. Choose “Erase all” if you want to delete all entries and press .
4. “Erase all?” will appear in the display and press  to confirm.

Handset volume

The handset volume can be adjusted during a call with the  and  push buttons on the front side of the telephone.

Additional amplification can be switched on by pressing the  button. When additional amplification is activated, the handset volume can be raised to up to 30 dB. If the amplification is switched on, the  button will light up.

The additional amplification will be switched off by pressing the  button again.

Switching the microphone off/on

You can switch the microphone on your telephone off and on during a telephone call.

Procedure for switching off the microphone:

1. Press the button . The microphone is now deactivated.
2. Press the button  again to switch the microphone back on.

HOLD

This function enables you to interrupt a conversation temporarily without ending the call.

Press the  button to interrupt the conversation temporarily.

Inserting a dialing pause (PABX)

Press the  button to interrupt dialing for 1 second.

Inquiry button (Recall time)

This function is required in order to activate certain T-Net features. If you press the  button, the telephone line is interrupted for a certain length of time.

Telephone book

Using the telephone book

Names and numbers can be stored in the telephone book. Once a telephone number is stored, you can call that number by pressing a minimum number of keys. Up to 30 telephone numbers can be stored in the telephone book. Each telephone number can contain up to 24 digits.

Storing name/number

To make new entries in the telephone book, proceed as follows:

1. Enter the desired telephone number on the keypad. Press  to delete the last digit entered.
2. Press . „Number:“ and the entered telephone number will be shown in the display.
3. Press  again and enter a desired name to the telephone number over the keypad.
4. Press the corresponding key as frequently as necessary to select the desired letter (see table below). Press  to delete the last letter entered.
5. Press  to confirm your name entry and switch to the selection of the ringtone for the entry.
6. Use  or  to page between the ringtones.
7. Press  to complete the programming process.

Key	Letter / character
0	0
1	[Space] _ * , 1
2	Aa Bb Cc (2
3	Dd Ee Ff) 3
4	Gg Hh Ii # 4
5	Jj Kk Ll / 5
6	Mm Nn Oo ' 6
7	Pp Qq Rr Ss 7
8	Tt Uu Vv ? 8
9	Ww Xx Yy Zz 9

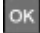
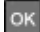
Note

If you want to shift the case between upper and lower, while entering letters, press the key .

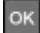
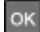
Placing a call with the telephone book

1. Press .
2. Use  or  to select the desired entry.
3. Lift up the handset or press either the  button or the  button to dial the number you have selected.

Editing an entry in the telephone book

1. Press . The first entry in the telephone book will be shown.
2. Use  or  to select the desired entry you would like to edit.
3. Press  after you have selected the desired entry. "Edit" will be shown in the telephone display.
4. Press  and the number of the entry will be shown on the telephone display.
5. Press  to delete a number or press the desired number on the keypad.
6. Press  to confirm your number entry and switch to the entry of the name.
7. Press  to delete a letter or press the corresponding key as frequently as necessary to select the desired letter (see table on page 36).
8. Press  to confirm your name entry and switch to the selection of the ringtone for the entry.
9. Use  or  to page between the ringtones.
10. Press  to complete the programming process.

Deleting an entry in the telephone book

1. Press . The first entry in the telephone book will be shown.
2. Use  or  to select the desired entry you would like to delete.
3. Press  after you have selected the desired entry. "Edit" will be shown in the telephone display.
4. Use  or  to select "Erase".
5. Press  and the selected entry will be deleted.

One-touch dial keys

Allocating telephone numbers to one-touch dial keys

The 4 speed-dial keys can be programmed with any telephone number.

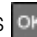
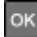
1. Enter the desired telephone number on the keypad. Press  to delete the last digit entered if necessary.
2. Press the desired one-touch dial key for longer than 3 seconds. „Number:“ and the entered telephone number will be shown in the display.
3. Press  again and enter a desired name to the telephone number over the keypad.
4. Press the corresponding key as frequently as necessary to select the desired letter (see table on page 36). Press  to delete the last letter entered if necessary.
5. Press  to confirm your name entry and switch to the selection of the ringtone for the entry.
6. Use  or  to page between the ringtones.
7. Press  to assign the selected number to the corresponding location.

Using the one-touch dial keys

Take the handset from the cradle or press either the  button or the  button.

If you press one of the keys marked  ...  the telephone number stored for that key will be dialed automatically.

Deleting one of the one-touch dial keys

1. Press . The first entry in the telephone book will be shown.
2. Use  or  to page to the desired one-touch dial entry in the phonebook. One-touch dial entries are marked with M1...M4.
3. Press .
4. „Edit“ will be shown in the telephone display.
5. Use  or  to page between „Edit“ and „Erase“.
6. Press  after you have chosen „Erase“.
7. The telephone number is deleted from the one-touch dial key and from the telephone book.

Calling number display

How it works

The number of the person calling you will appear in the telephone display. The telephone supports both of the current systems (DTMF/ FSK). Your telephone will store the last 20 calling numbers automatically. If the memory is full, the oldest entry will be deleted.

Important!

Calling number display will only work if this service is supported by your telecom supplier. Please inquire with your telephone company.

Reading and calling entries

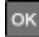
The last 20 telephone numbers recognized will be automatically stored in the telephone.

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Use ▼ or ▲ to page between the entries. The first 12 digits of the telephone number or a stored name will appear in the display. Additionally the date and the time of the call record will be shown.
3. Once the desired telephone number appears in the display, lift the handset or press either the  button or the  button. The number you have selected will be dialed.

Note:

The handset must be resting on the cradle when you select the telephone number.

Storing an entry

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Use ▼ or ▲ to page between the entries.
3. Press  if the desired entry will be shown. "Store" will be shown in the telephone display.
4. Press , „Number:" and the telephone number will be shown in the display.
5. Press  again and enter the desired name of the telephone number on the keypad.
6. Press the corresponding key as frequently as necessary to select the desired letter (see table on page 36). Press  to delete the last letter entered.
7. Press  to confirm your name entry and switch to the selection of the ringtone for the entry.
8. Use ▼ or ▲ to page between the ringtones.
9. Press  to complete the programming process.

Deleting an entry

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Use ▼ or ▲ to page between the entries.
3. Press **OK** if the desired entry will be shown.
“Store” will be shown in the display.
4. Use ▼ or ▲ to page between „Store“, „Erase“ and „Erase all“.
5. Choose “Erase” and press **OK** to delete the chosen entry.

Deleting all entries

1. Press ▼ or ▲ to enter the calling number display. The last entry will be shown in the display.
2. Press **OK**. “Store” will be shown in the display.
3. Use ▼ or ▲ to page between „Store“, „Erase“ and „Erase all“.
4. Choose “Erase all” and press **OK**. “Erase all?” will be shown in the display.
5. Press **OK** again to confirm and delete all entries.

Settings

Language - Date format - Time format - Date & time - Ring volume - Ring melody

Setting the language

1. Press . „Set Display“ will be shown in the display.
2. Press  again. „Set Language“ will be shown in the display.
3. Press  to enter the language selection.
4. Use ▼ or ▲ to page to the desired language.
5. Press  to confirm your selection.

Setting the date format

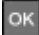
1. Press . „Set Display“ will be shown in the display.
2. Press  again to enter the display setting menu.
3. Use ▼ or ▲ to page to “Date Format”.
4. Press  to enter the date format selection.
5. Use ▼ or ▲ to page between “MM-DD” (MonthMonth-DayDay) and „DD-MM“ (DayDay-MonthMonth).
6. Press  to confirm your selection.

Setting the time format

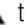
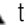
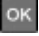
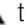
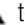
1. Press . „Set Display“ will be shown in the display.
2. Press  again to enter the display setting menu.
3. Use ▼ or ▲ to page to “Hour Format”.
4. Press  to enter the time format selection.
5. Use ▼ or ▲ to page between “12hour clock” and “24hour clock”.
6. Press  to confirm your selection.

Setting the date & time

1. Press . „Set Display“ will be shown in the display.
2. Use ▼ or ▲ to page to “Date & Time”.
3. Press  to enter the date and time menu. “Time:” will be shown in the display and the first number of the hours blinks.
4. Use the keypad to enter the hours.
5. The telephone switches to the minutes after the enter of the hours and the first digit blinks.
6. Use the keypad to enter the minutes.
7. Press . The telephone switches to the date entry and the first digit blinks.

8. Use the keypad to enter the numbers for the date. It depends on the setting of the date format if you start with the day and followed by the month or vice versa. The telephone switches to the next digit after each keypad entry.
9. Press . „YEAR:” and the set number of the year will be shown on the display. Additionally the third digit blinks.
10. Use the keypad to enter the desired number of the year.
11. Press  to confirm your entries.

Setting the ring volume

1. Press . „Set Display” will be shown in the display.
2. Use  or  to page to „Set Phone”.
3. Press  to enter the phone setting menu. „Set Ring” will be shown in the display.
4. Press . „Ringer volu.” will be shown in the display.
5. Press  to enter the ring volume selection.
6. Use  or  to page between OFF and 5 volume levels.
7. Press  to confirm your selection.

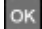
Setting the ring melody

1. Press . „Set Display” will be shown in the display.
2. Use  or  to page to „Set Phone”.
3. Press  to enter the phone setting menu. „Set Ring” will be shown in the display.
4. Press . „Ringer volu.” will be shown in the display.
5. Use  or  to page to „Ringer tone”.
6. Press  to enter the ring melody selection. The set ring tone will be shown.
7. Use  or  to page between 10 different melodies. You hear the ring tone.
8. Press  to confirm your selection.

Settings

Key tone - Contrast - Recall time (Flash)

Setting the key tone

1. Press . „Set Display” will be shown in the display.
2. Use  or  to page to „Set Phone”.
3. Press  to enter the phone setting menu. “Set Ring” will be shown in the display.
4. Press . „Ringer volu.” will be shown in the display.
5. Use  or  to page to „Keypad Tones”.
6. Press  to enter the key tone menu.
7. Use  or  to page between “ON” and “OFF”.
8. Press  to confirm your selection.

Setting the contrast

1. Press . „Set Display” will be shown in the display.
2. Press  again. „Set Language” will be shown in the display.
3. Use  or  to page to “Set Contrast”.
4. Press  to enter the contrast selection. The set contrast level will be shown.
5. Use  or  to page between 8 different contrast levels.
6. Press  to confirm your selection.

Setting the recall time (Flash)

1. Press . „Set Display” will be shown in the display.
2. Use  or  to page to „Set Phone”.
3. Press  to enter the phone setting menu. “Set Ring” will be shown in the display.
4. Use  or  to page to „Set Flash”.
5. Press  to enter the recall time selection. „300ms” will be shown in the display.
6. Use  or  to page between 100ms, 300ms, and 600ms.
7. Press  to confirm your selection.

Settings

Dynamic font - Dialing mode - Call waiting

Setting the dynamic font

1. Press . „Set Display” will be shown in the display.
2. Press  again to enter the display setting menu.
3. Use  or  to page to “Dim:X--->x”.
4. Press  to enter the dynamic font menu.
5. Use  or  to page between “ON” and “OFF”.
6. Press  after you have chosen the desired setting.

Setting the dialing mode

1. Press . „Set Display” will be shown in the display.
2. Use  or  to page to „Set Phone”.
3. Press  to enter the phone setting menu. “Set Ring” will be shown in the display.
4. Use  or  to page to „Dialing Mode”.
5. Press  to enter the dialing mode setting menu.
6. Use  or  to page between “Tone” and “Pulse”.
7. Press  after you have chosen the desired setting.

Setting the call waiting

1. Press . „Set Display” will be shown in the display.
2. Use  or  to page to „Set Phone”.
3. Press  to enter the phone setting menu. “Set Ring” will be shown in the display.
4. Use  or  to page to “Call waiting”.
5. Press  to enter the call waiting setting menu.
6. Use  or  to page between “ON” and “OFF”.
7. Press  after you have chosen the desired setting.

Settings

Auto On Hook - Auto Amp - Mic Amp

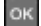
Setting the Auto On Hook

1. Press . „Set Display” will be shown in the display.
2. Use ▼ or ▲ to page to „Set Phone”.
3. Press  to enter the phone setting menu. “Set Ring” will be shown in the display.
4. Use ▼ or ▲ to page to “AUTO ON HOOK”.
5. Press  to enter the auto on hook setting menu.
6. Use ▼ or ▲ to page between “ON” and “OFF”.
7. Press  after you have chosen the desired setting.

Setting the Auto Amp function

1. Press . “Set Display” will be shown in the display.
2. Use ▼ or ▲ to page to “Auto AMP”.
3. Press  to enter the Automatic amplification menu. “OFF” will be shown in the display.
4. Use ▼ or ▲ to page between “ON” and “OFF”
5. Press  after you have chosen the desired setting.

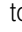
Setting the Mic Amp function

1. Press . “Set Display” will be shown in the display.
2. Use ▼ or ▲ to page to “Mic AMP”.
3. Press  to enter the Microphone amplification menu. “0dB” will be shown in the display.
4. Use ▼ or ▲ to page between “+4dB”, “0dB” and “-4dB”.
5. Press  after you have chosen the desired setting.

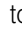
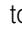
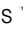
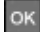
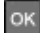
Bluetooth

For information on your Bluetooth headset or any other Bluetooth receiver device, please consult the respective manufacturer's manual.

Pairing of a Bluetooth headset

1. Press . "Set Display" will be shown in the display.
2. Use  or  to page to "Bluetooth".
3. Press  to enter the "Bluetooth" menu.
"Add device" will be shown in the display.
4. Press . "Please wait." will be shown in the display.
5. Start the pairing procedure of your Bluetooth headset or other Bluetooth-enabled receiver device. Please refer to the device's according manual.
6. After a short while, the name of the respective Bluetooth receiver will appear on the display.
7. Press  and please wait 3 seconds.
8. A confirmation tone is given by the telephone. Both units have been successfully paired and the -icon is displayed on the screen.

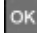
Connecting of a Bluetooth headset

1. Press . "Set Display" will be shown in the display.
2. Use  or  to page to "Bluetooth".
3. Press  to enter the "Bluetooth" menu.
"Add device" will be shown in the display.
4. Use  or  to page to "Device list".
5. Press . The name of the first Bluetooth headset in the list is displayed.
6. Browse the paired devices by using the buttons  or .
7. Press  when the desired device is displayed. "Connect" will be shown in the display.
8. Press  to connect with the selected device.

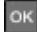
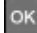
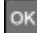
Note:

In order to connect a Bluetooth headset or any other Bluetooth receiving device with the telephone, the device has to be switched on.

Disconnecting of a Bluetooth headset

1. Press . “Set Display” will be shown in the display.
2. Use  or  to page to “Bluetooth”.
3. Press  to enter the “Bluetooth” menu.
“Add device” will be shown in the display.
4. Use  or  to page to “Device list”.
5. Press . The name of the first Bluetooth headset in the list is displayed.
6. Browse the paired devices by using the buttons  or .
7. Press  when the desired device is displayed. “Connect” will be shown in the display.
8. Use  or  to page to “Disconnect”.
9. Press  to disconnect with the selected device.

Removing of a paired headset

1. Press . “Set Display” will be shown in the display.
2. Use  or  to page to “Bluetooth”.
3. Press  to enter the “Bluetooth” menu.
“Add device” will be shown in the display.
4. Use  or  to page to “Device list”.
5. Press . The name of the first Bluetooth headset in the list is displayed.
6. Browse the paired devices by using the buttons  or .
7. Press  when the desired device is displayed. “Connect” will be shown in the display.
8. Use  or  to page to “Remove”.
9. Press  to remove the selected device from the list.

Additional ports

Connection jack for vibrating pillow and/or modules

Your telephone is equipped with a connection jack for a vibrating pillow or other additional signaling modules. You can take advantage of this option to signal an incoming call not only with a very loud ring and the very bright flashing light, but also with an optionally available vibrating pillow and/ or other module.

The following optional modules can be connected to the telephone:

- Vibrating pillow
- Flash module
- Switch module
- Acoustic module

Headset port

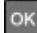
Standard headset port with a 2.5 mm jack.

Headphone port

3.5mm output for connecting headphones, an induction loop or such like.

Resetting the telephone to the factory settings

Setting the telephone to the default settings

1. Press . „Set Display” will be shown in the display.
2. Enter 0,4,2,2,0,0,0,1,1 over the keypad
3. Press . You hear a signal tone.
4. Disconnect the small PSU plug from the DC connection jack at the backside of the telephone.
5. Please wait three seconds and reconnect the small PSU plug with the DC connection jack.
6. The telephone is set to the default settings now.

Default settings:

One-touch dial keys:	Empty
Redial memory:	Empty
Calling number display memory (CLIP):	Empty
Telephone book:	Empty
Recall time:	300 ms
Dialing mode:	Tone
Ring tone:	2
Keypad tones:	Switched on
Volume:	Level 4

Power failure operation

Should there be a power failure, you can still use your telephone.

If you would like to use the display function during a power failure, you can put 4 batteries (mignon or AA) into the telephone.

Note:

The dead-battery symbol  will appear in the display if the batteries are dead.

If your telephone doesn't work...

General information

Make sure that the telephone cable isn't damaged and is correctly inserted. Disconnect any additional equipment, extension cables and other telephones. If the telephone works then, this indicates that the fault is caused by another piece of equipment. Try to connect your telephone to a neighbor's line, for example. If it works there, this indicates that there may be a fault on your telephone line. Call the service provider to report the fault.

No number appears when the phone rings

- Calling number display is a function that has to be supported by your service provider.
- If no calling number appears in the display, this means that no calling number information is available or that the caller has an unlisted number. This can also happen when a call is received from a telephone exchange.
- If the telephone is connected to an extension, the calling number display function will not usually work.

The telephone doesn't work

- Make sure that the mains adapter is correctly inserted into the telephone and into the mains socket.
- Make sure that the telephone cable is correctly inserted into the telephone and into the telephone socket.
- Connect another telephone to your telephone line. If it works, it indicates that your telephone is probably faulty.
- Only use the original cable or a telephone cable with an international configuration.

Appendix

Maintenance and care

The telephone is maintenance-free. If the telephone becomes dirty, wipe it occasionally with a soft, damp cloth. Never use alcohol, thinner or other organic solvents.

The telephone should never be exposed to sunlight for a long time and should also be protected from intense heat, moisture or strong mechanical vibrations.

Warranty

The telephone is a very reliable product. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly. This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away. The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt. **Always specify the product number in any event.**

Environmental note/ Waste disposal

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and other European countries with a separate collection system for these devices).

The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment. By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

Technical data

Power supply:

Plug-in PSU:

Primary: 100-240 V 50 Hz / 0.2A Max -

Secondary: 7.5 V DC / 300 mA

Dimensions (WxDxH):

178 mm x 210 mm x 96 mm

Weight:

900 g

CE symbol

This telephone satisfies the following EU directives:

- 2011/65/EC RoHS directive
- 2002/96/EC WEEE directive
- 1999/5/EC R&TTE directive
- 2004/108/EC EMC directive
- 2006/95/EC Low voltage directive

Compliance with the directives listed above is confirmed by the CE seal on the telephone. CE compliance declarations are available on the Internet at www.humantechnik.com.

Hereby, Humantechnik GmbH declares that this device is in compliance with the essential requirements and other relevant provisions of Directive 1999/5/EC. For the declaration of conformity, visit the Web site www.humantechnik.com

CE 0700

Technical specifications subject to change without notice.

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Sarabec Ltd