*free***TEL II/S**

D

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Seite 2

Anrufbeantworter

GB

Operation Instructions

Page 8

Answering machine

F

Mode d'emploi

Page 14

Répondeur

NL

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Pagina 20

Antwoordapparaat

I

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Pagina 26

Segreteria telefonica

E

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Página 32

Contestador automático

Allgemein

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General


With the purchase of the *freeTEL II/S*, you have decided to buy a telephone with an integrated answering machine. In order to activate this answering machine and proper use, please read the following explanations.

Answering machine settings


Recording an outgoing message

Attention:


Please speak loudly and clearly into the microphone when recording a message.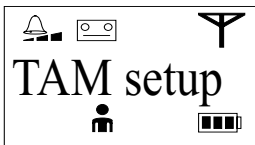

1. Use the buttons ▲▼ to select the "TAM setup" menu option and press the button ↩.

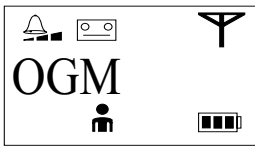
2. Now scroll to the "OGM" option and press the button → to confirm.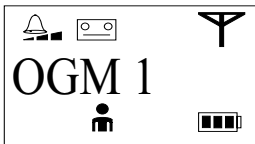


3. You can choose the 1st and 2nd outgoing messages.
 - a. The 1st outgoing message is now played in "ANS/2Ring" or "ANS/6Ring" mode.
 - b. The 2nd outgoing message is only played back in "Ans only" mode.


4. Once you have selected the required outgoing message, press the button ↩.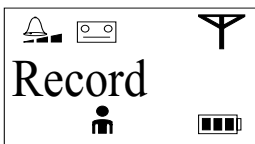

5. Now select the "Record" menu option and press the button ↩. "Wait.." will appear in the display.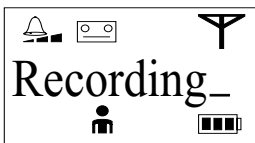

6. As soon as "Recording_" appears in the display, you can record your outgoing message.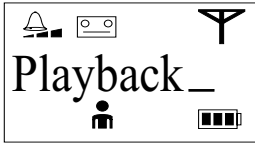


- Once you have ended your outgoing message text, press the button . The dictated outgoing message text is then played automatically, and "Playback_" appears on the display.
- Press **C** to stop playing the outgoing message.
- If you want to change the spoken text, retry the upon described points "Record". The already recorded text will be erased.


Note:

The 1st outgoing message is only played in "Answer/Record" mode, and the 2nd outgoing message is only played in "OGM only" mode.


Listening to an outgoing message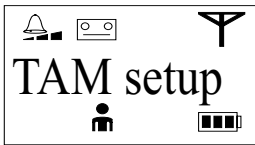


- Use the buttons to select the "TAM setup" menu option and press the button .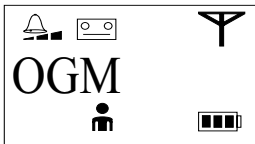


- Now scroll to the "OGM" option and press the button to confirm.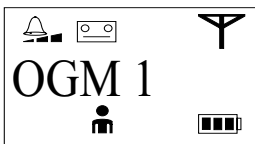

- Use to select the outgoing message you want to listen to and press .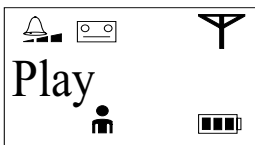

- Now use the buttons to select the "Play" menu option and then press the button .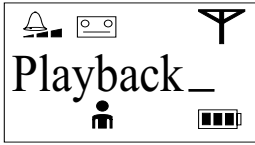

5. "Playback_" appears on the display. You hear the outgoing message which was dictated.
6. Press **C** to stop playback.


Switching on the answering machine (TAM)

Please note: An outgoing message must be recorded on the machine before you can make one of these settings.


You have a choice of 4 different settings:

- Ans Off The answering machine is switched off. The telephone doesn't answer calls.
- Ann Only The telephone only plays an outgoing message when it receives a call. The caller is not able to leave a message.
- Ans/2Ring The answering machine switches on after 2 rings and is ready to record information.
- Ans/6Ring The answering machine switches on after 6 rings.


Procedure for defining the "ANS/2Ring" setting, for example: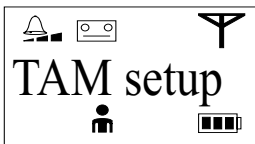

1. Use the buttons ▲▼ to select the "TAM setup" menu option and press the button ↵.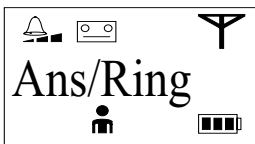

2. Now scroll to the "Ans/Ring" option and press the button ↵ to confirm.

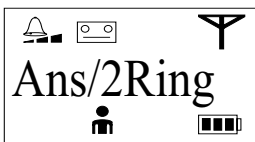
3. Now use the buttons ▲▼ to select the required function.
4. Press the button ↵ to confirm. Once the answering machine is switched on, the icon ☎ lights up on the display.

Messages


General

A red light will flash on the cordless phone if there are any new messages.


Playing a message

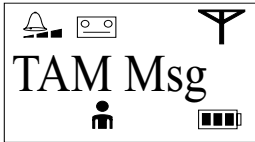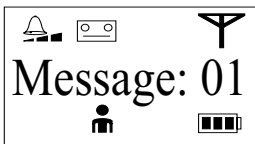
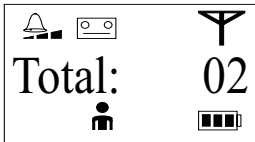
1. Press the button  once. Then select the “TAM Msg” menu option.

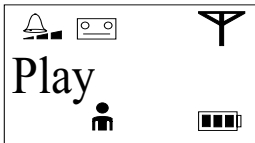2. Press the button  to confirm. “Play” appears on the display.

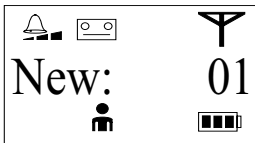3. Press the button  once again. The number of new messages and the number of messages stored are displayed.


4. The messages are then played. Press numerical key **8** to switch between the number of the message and the day and the time of the recording.


Important control elements in the messages menu

- a. Press numerical key **1** once to start a new message.
- b. Press numerical key **1** a second time to start the previous message.
- c. Press numerical key **3** to go to the next message.
- d. To delete a message, press numerical key **4** and then the button  to confirm.
- e. Press **C** to stop playing the messages.
- f. Press numerical key **8** to switch between the number of the message and the day/time of the recording.

Deleting all old messages

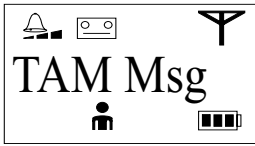
1. Press the button ▲ once to get to the "TAM Msg" menu option.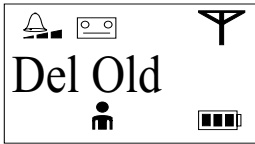

2. Then press the button ⇨.


3. Use the buttons ▲▼ to select the "Del Old" menu option and press the button ⇨ to confirm.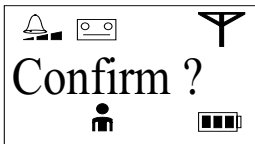

4. When "Confirm?" appears on the display, press the button ⇨ to confirm.


Recording a memo

You can record a memo which is played as a normal message.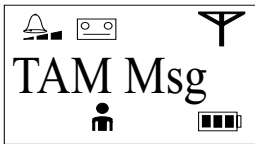


1. Press the button ▲ once to get to the "TAM Msg" menu option.

2. Then press the button ⇨.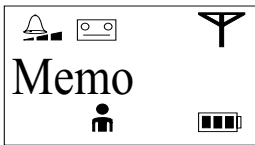

3. Use the buttons ▲▼ to select the "Memo" menu option and press the button ⇨.

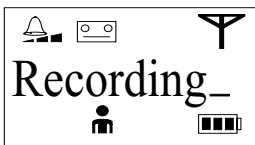4. "Recording_" appears on the display. You can now dictate your memo.

5. Press **6** to end the recording.

6. In order to listen to a memo, follow the procedure described under "Playing a message".

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Humantechnik GHL AG