HUMANTECHNIK

*free***TEL III**

GB

Operation Instructions

Page 2

Cordless telephone
with amplifier

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Safety instructions

In order to be able to operate and use your telephone safely, please read the following safety instructions and the section on “Commissioning”.

- Position the base station on a non-slip surface.
- Lay the connecting cable accident-proof.
- Never expose the telephone to the following sources of influence:
 - heaters
 - direct sunlight
 - moisture
 - other electrical devices
- Protect your telephone from damp, dust, aggressive fluids and vapors.
- Plug the connection cord into the corresponding jack.
- Only connect approved accessories.
- Never open the base station or handset yourself!

Contents	page		
Commissioning		View name/number of a contact entry	14
Packaging content	5	Changing the name/number of an entry in the phone book	14
Telephone connection	5	Storing a telephone number from the redial	15
Batteries	6	Delete name/number in phone book	15
Battery charge status display	6	Delete all entries in phone book	15
Range	6		
Paging the handset	6		
Controls		Call number display	
Handset controls	7	General	16
Base station controls	8	How it works	16
Softkeys	9	Reading and calling entries	17
Display	9	Saving an entry in the phone book	17
Date/Time setup	10	Delete entry from the telephone number list	17
Using the telephone		Delete all entries from the telephone number list	17
Making calls	10	Various telephone number lists	
Terminate dialling	10	Telephone number list of incoming calls	18
Answering calls	10	Telephone number list of outgoing calls	18
Using the handsfree function during a call	11	Answering machine settings	
Redial	11	Recording an outgoing message	19
Setting the speaker volume/treble/bass control	11	Listening to an outgoing message	19
Switch microphone amplification on/off	11	Switching on the answering machine (TAM)	20
Call length display	12	Set remote query PIN	20
Switching the microphone on/off	12	Messages	
Correction of wrong inputs	12	General	21
Phone book		Playing a message	21
Using the phone book	12	Deleting all old messages	21
Button configuration	12	Recording a memo	21
Save first entry in phone book	13		
Storing a name/number	13		
Calling from the phone book	13		

continued:

Answering machine on the go (remote control)

General	22
Start remote control	22
Listen to messages and manage	22
Listen to announcement 1 or record again	23
Listen to announcement 2 or record again	23
Recording a memo	23
Set answering machine mode	23

Melody/Handset ringer volume

Setup

Changing the menu language	24
Automatic call acceptance	24
Set contrast	25
Activate/deactivate key tones	25
Changing the handset name	25
Change menu colour	26
Activate/deactivate time	26
Change background colour of splash screen	26
Change background colour of menu lists	27
Register a handset	27
Register other handsets	28
Deleting a handset from the base station	28
Dial pause setup	28
Flash time setup	29
Dial mode setup	29
Key lock	29
Disabling the permanent RF radiation	30

Use several handsets

Internal calls	31
Transferring calls between handsets	31

Bluetooth®

Activate/deactivate Bluetooth® function	31
Couple Bluetooth® devices	32

Connecting special accessories

Troubleshooting

Resetting the telephone to the factory settings

Important notice

Appendix

Maintenance and care	34
Warranty	34
Environmental note/	
Waste disposal	34
Rechargeable batteries	34
Technical Data	35
CE symbol	35

Commissioning

Packaging content

Please check if all following components are included:

- Telephone (handset)
- Base station
- Power supply unit
- Telephone cable
- 3 batteries (NiMH)
- Operating instructions
- Warranty card

If any parts are missing, please immediately contact your dealer or the manufacturer directly.

Telephone connection

1. Connect the power supply unit to the socket and to the jack on the base station. (see figure 1).
2. Insert the plug of the telephone cable into the base station. (see figure 1).
3. Plug the telephone cable into the telephone socket for your telephone line. (see figure 2).
4. Place the two batteries into the handset and replace the battery cover. (see figure 3). Please ensure that the batteries are inserted properly!
5. Place the handset and keypad facing forward into the base station. (see figure 2). Once the handset is correctly inserted, the battery display  starts to flash on the telephone display and the red charge LED lights on the base station.


Figure 1

Figure 2

Figure 3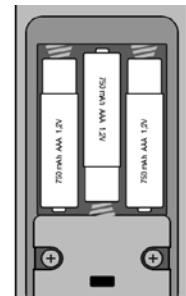

Attention!

Don't position the base station in close proximity to, for instance, televisions, VDUs, other telephones, fans, etc. in order to minimize the risk of interference. Don't expose the base station to direct sunlight or position it close to a heater.

Batteries

The telephone comes with NiMH batteries. If the batteries are correctly charged and discharged, they will work for a long time. Fully charged batteries have a standby time of approx. 100 hours or a maximum talk time of 10 hours. Discharged batteries take approx. 12 hours to recharge. The battery charge status indicator flashes while the telephone is being charged. Once charging is finished, the symbol  lights up on the display.

The telephone possesses automatic charging control, which prevents batteries from becoming over-charged.


Attention!

The batteries have to be charged for 12 hours the first time the telephone is connected.

Note:

Avoid leaving the batteries stowed in the handset if your telephone is out of use for a lengthy period of time (more than 6 months).

Battery charge status display
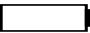
When the batteries are virtually empty, the icon  appears on the display of the handset, and a warning signal is sounded. If the telephone is not charged now, the handset switches off automatically due to low battery. Personal Preferences (phonebook, redial lists, etc.) are preserved, even if the batteries are discharged or removed.

Range

The telephone's radio range can vary between 50 and 300 m. The radio signals for transmission can be disrupted by a variety of factors. The maximum range is only achieved if there are no obstacles between the telephone and the base station (under ideal conditions the range is 300 m). The range is limited by walls inside a house or apartment. Ideally, the base station should be placed in a slightly raised position. If the handset is taken outside the range during a call, a warning signal sounds. If the handset can't be brought closer to the base station, you may lose the call connection.

Paging the handset

If you have mislaid your handset again, please use the paging feature.

1. Press the button  on the base station.
2. All the handsets which have been added are called.
3. The call ends when any key is pressed on one of the cordless phones being paged.

Controls

Handset controls




▲ ▼ Scroll: ▲ up / Message
▼ down / Call number display


◀ ▶ Scroll: ◀ left / Microphone On/Off
▶ right / Internal calls

☎ Return to the start menu in the event of an input error

Base station controls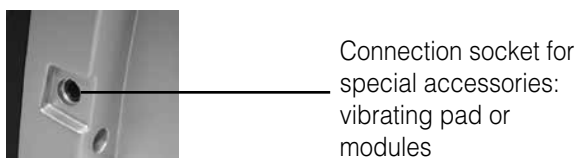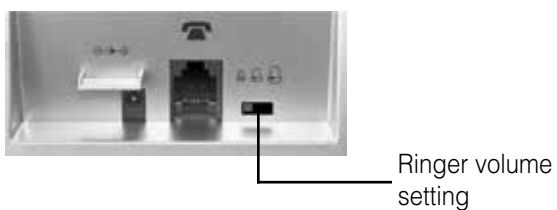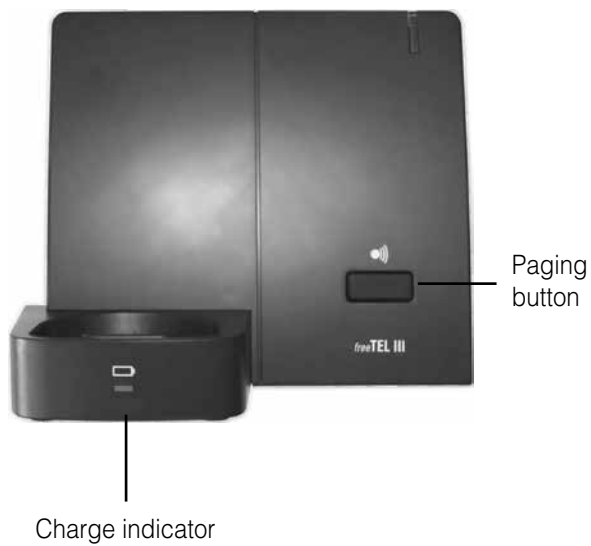

Softkeys

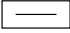The two softkeys  left and right from the control key are assigned with different functions in different menus. In this user manual they are always named corresponding to the texts in the display.

Button

(left)

In stand-by mode

- Open main menu

In the menu

- Diverse functions


(right)


- Open phonebook

- Diverse functions

Display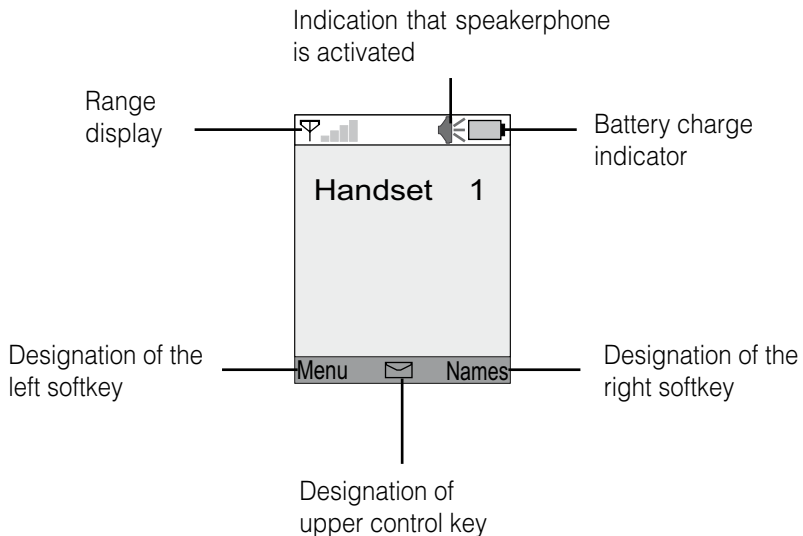

Date/Time setup

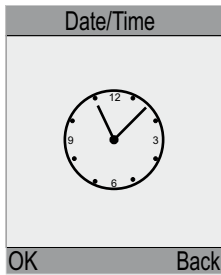Press **Menu** (left softkey) → navigate to **Date/Time** using key ▼ or ▲ → Press **OK** (left softkey)

Enter day, month and year in 8 digits (e.g. 1 2 0 5 2 0 1 1 for 12.05.2011). You can jump between the entry fields using the keys ◀ and ▶. After finalizing the date entry the cursor automatically jumps to the first digit of the time entry fields.

Enter hours and minutes in 4 digits (e.g. 0 9 2 5 for 09:25 hours). You can jump between the entry fields using the keys ◀ and ▶. Press **OK** (left softkey) to store your settings or select with the keys ▲ ▼ the menu item **Auto update**. Now activate or deactivate the automatic time adoption from the telephone line using the keys ◀ or ▶. Press **OK** (left softkey) to save your settings.

Using the telephone

Making calls

1.  Enter the required telephone number (max. 20 digits). If you enter a wrong digit, press  to clear it.
2. Press . The number is dialed.
3. End the call by pressing the  button again.

Terminate dialling

To terminate the dialling process, press the key  or place the handset back into the base station or desktop charger.


Answering calls

An incoming call is signalled several times at the handset:

- acoustically, with a ringtone
- visually, by means of a light signal at the phone and notification on the display
- mechanically by means of vibrations.

1. The telephone rings.
2. Press  to answer the call.
3. End the call by pressing the  button.

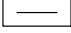Using the handsfree function during a call

1. Press the left softkey () or the key  to activate the hands-free mode during a call.
2. To deactivate the hands-free, press the left softkey () or the key .

Note: If the speakerphone is switched on,  will appear in the display.

Redial

Press **LNR/II** to display the last telephone numbers dialed.

1. Press the **LNR/II** button and then scroll to the number you require using the   buttons.
2. Press the  button to dial the number.

Setting the speaker volume/treble/bass control

The speaker volume can be set during a telephone call using the arrow buttons on the front of the handset.

The speaker volume can be increased or lowered in five 4 dB increments by pressing the   buttons.

Press  to activate additional amplification (key lights up in red) and to activate tone control.

You can also raise treble or bass tones in increments of 10 dB using the tone control (slide switch on the left side of the housing).

Press  again to deactivate additional amplification and tone control (key is no longer lit up).

Note:

The amplification switch only works when the receiver is activated.

Switch microphone amplification on/off

The »freeTEL III« is equipped with an additional microphone amplification, which can be switched on as needed. This makes it possible for you to raise the volume of quiet sound transmissions to enable the person on the other end to understand better. To switch on microphone amplification, move the sliding switch **M** to .

Call length display

The current call time is shown on the display during a call. After terminating the call „**Call Ended**“ as well as the duration of the call is displayed.

Switching the microphone on/off


You can switch the microphone on your handset on and off during a telephone call. Procedure for switching off the microphone:

1. Press the ◀ button. The microphone is now deactivated and the 📞 symbol is shown on the display.
2. Press the ◀ button again to switch the microphone back on.

Correction of wrong inputs

- Navigate to the wrong entry using the keys ◀ and ▶ if a blinking cursor is shown on the display.
- Use **Clear** (right softkey) to delete the character/digit left of the cursor.
- Insert new character/new digit left of the cursor.
- When entering date and time etc. overwrite character with dark background.

Phone book

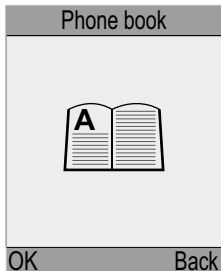Using the phone book

Up to 100 entries can be stored with a name and telephone number in the phone book. Once a telephone number has been stored, the call can be placed by pressing just a few keys. Each name can be up to 12 characters in length, and the number can be up to 20 digits long.

Button configuration

Button	Letter / Symbol	Button	Letter / Symbol
1	[Space] 1	6	M N O m n o 6 Ö ö ó ò ô ñ ñ
2	A B C a b c 2 Å Ä Á å ä á à ç	7	P Q R S p q r s 7 ß
3	D E F d e f 3 Ě Ě Ě Ě ě ě ě	8	T U V t u v 8 Ů Ů Ů Ů ů ů ů
4	G H I g h i 4 í î î	9	W X Y Z w x y z 9 Ÿ Ý ý ý
5	J K L j k l 5	0	0

Save first entry in phone book

Press **Names** (right softkey)

-  Enter the required name (max. 12 digits).
- Press the ▼ button.
-  Enter the required telephone number (max. 20 digits).
- Press **OK** (left softkey).

Storing a name/number

Press **Menu** (left softkey) → **Phone book** → Press **OK** (left softkey).

- Press the ▼ button. → **New entry** → Press **OK** (left softkey).
-  Enter the required name (max. 12 digits).
- Press the ▼ button.
- Enter the required telephone number (max. 20 digits).
- Press **OK** (left softkey).

Calling from the phone book

You do have two possibilities, to make a call directly from the phone book:

1. Possibility:

1. Press **Names** (right softkey).
2. Select the desired entry using the keys ▲ ▼ or use the search function. If you are looking for an entry starting with a W, for instance, press number 9 (»W«).
3. Press the key  to retrieve the number of the desired contact.

Note:

At least one entry has to be stored in the telephone directory in order to select or call an entry directly via **Names** (right softkey). If this is not the case you can now enter a contact with name and number.

2. Possibility:

Press **Menu** (left softkey) → **Phone book**

- Press **OK** (left softkey)
- **Find** → Press **OK** (left softkey)
- Select the desired entry using the keys ▲ ▼ or use the search function. If you are looking for an entry starting with a W, for instance, press number 9 (»W«).
- Press the key  to retrieve the number of the desired contact.

Note: You can abort the search process by pressing the  button.

View name/number of a contact entry

Press **Names** (right Softkey)

→ Select the desired entry with the keys ▲ ▼.

→ Press **Option** (left softkey)

→ **View** → Press **OK** (left softkey)

→ Name and telephone number are shown on the display.

→ Press **Back** (right softkey) to return to the contact list.

→ Select new entry with the keys ▲ ▼.

or: → Press  button to abort the search process.

There is another possibility to view a phone book entry. Follow the procedure shown below:

Press **Menu** (left softkey) → **Phone book**

→ Press **OK** (left softkey)

→ **Find** → Press **OK** (left softkey)

→ Select the desired entry with the keys ▲ ▼.

→ Press **Option** (left softkey) → **View** → Press **OK** (left softkey).

Changing the name/number of an entry in the phone book

Press **Names** (right Softkey)

→ Select the desired entry with the keys ▲ ▼.

→ Press **Option** (left softkey).

→ **Edit** → Press **OK** (left softkey).

→ Navigate to the desired position in the name using the keys ◀ and ▶ when the blinking cursor is shown on the display.

→ Delete the character right of the cursor using **Clear** (left softkey) or insert a new character using the keypad.

→ Press the ▼ button. → Navigate to the desired position in the number using the keys ◀ and ▶.

→ Delete the digit left of the cursor using **Clear** or insert a new digit using the keypad.

→ Press **OK** (left softkey) to save the changes.

There is another possibility to change a phone book entry. Follow the procedure shown below:

Press **Menu** (left softkey). → **Phone book**

→ Press **OK** (left softkey).

→ **Find** → Press **OK** (left softkey).

→ Select the desired entry with the keys ▲ ▼.

→ Press **Option** (left softkey). → **Edit** → Press **OK** (left softkey)

Storing a telephone number from the redial

- Press the **LNR//** key to view the redial list.
- Select the desired entry with the keys **▲ ▼**.
 - Press **Option** (left softkey).
 - **Store** → Press **OK** (left softkey)
 - **RECALL** Enter the required name (max. 12 digits).
 - Press **▼** button to navigate to the number.
- or directly:**
- Press **OK** (left softkey) to save the entry.

Additionally you can select further functions:

LNR// → Desired entry → Press **Option** (left softkey). →

Delete:

Delete desired entry

Delete all:

Delete **all** entries from the redial list.

Delete name/number in phone book

- Press **Names** (right softkey).
- Select the desired entry with the keys **▲ ▼**.
 - Press **Option** (left softkey). → **Delete**
 - Press **OK** (left softkey).
 - **?** appears on the display. → Press **OK** (left softkey) to delete the entry.

There is another possibility to delete a phone book entry. Follow the procedure shown below:

- Press **Menu** (left softkey) → **Phone book**
- Press **OK** (left softkey)
 - **Find** → Press **OK** (left softkey)
 - Select the desired entry with the keys **▲ ▼**.
 - Press **Option** (left softkey). → **Delete** → Press **OK** (left softkey).

Delete all entries in phone book

- Press **Names** (right softkey).
- Press **Option** (left softkey). → **Delete all**
 - Press **OK** (left softkey).
 - **?** appears on the display. → Press **OK** (left softkey) to delete all entries.

There is another possibility to delete all phone book entries. Follow the procedure shown below:

- Press **Menu** (left softkey) → **Phone book** → Press **OK** (left softkey)
 - **Find** → Press **OK** (left softkey)
 - Press **Option** (left softkey). → **Delete all** → Press **OK** (left softkey)
 - **?** appears on the display. → Press **OK** (left softkey) to delete all entries.
-

Call number display

General

During a call the number of the caller is shown on the display (caller ID) if the following conditions are met:

1. Your network provider supports CLI and CLIP:
 - CLI (Calling Line Identification):
The number of the caller is transmitted.
 - CLIP (Calling Line Identification Presentation):
The number of the caller is displayed.
2. You have applied for CLIP at your network provider.
3. The caller has applied for CLI at his network provider.

How it works

During a call the number or, if stored in the telephone directory, the name of the caller is shown on the display.

Conditions:

1. You have applied at your network provider for the transfer of the caller number (CLIP) to your display.
2. The caller has applied at this network provider that his number is transmitted (CLI). There will be "**Unavailable**" if the caller ID was not ordered by you or "**Private**", the caller ID is suppressed by the caller or if it was not ordered.

Note:

Not accepted calls will be shown as "**New calls**" on the display. This notification disappears if all numbers in the call list are viewed.

Not accepted calls can be indicated by the flashing of the white LED on the handset (a short flash every 8 seconds). This function is deactivated in default state.

To activate this function, please proceed as follows:

Press softkey **Menu** → **Setup**

→ Press softkey **OK** → **Handset**

→ Press softkey **OK** → **Missed calls**

→ Press softkey **OK**

→ Use the ◀ and ▶ buttons to select **“ON”** or **„OFF“**

→ Press softkey **OK** to save your settings.

Reading and calling entries

Press the ▼ button. → The last call will be shown at first entry in the call list.

→ Navigate through the entries using the keys ▲ ▼.

→ Press the  button to dial the number.

Note: Calls that have not been viewed yet are marked with „New call“ in the list.

Saving an entry in the phone book

Press the ▼ button. → The last call will be shown at first entry in the call list.

→ Navigate through the entries using the keys ▲ ▼.

→ Press **Option** (left softkey).

→ **Store** → Press **OK** (left softkey).

→  Enter the required name (max. 12 digits).

→ Press the ▼ button to navigate to the number.

or directly:

→ Press **OK** (left softkey) to save the changes.

Delete entry from the telephone number list

Press the ▼ button. → The last call will be shown at first entry in the call list.

→ Navigate through the entries using the keys ▲ ▼.

→ Press **Option** (left softkey).

→ **Delete** → Press **OK** (left softkey).

→  appears on the display.

→ Press **OK** (left softkey) to delete the entry.

Delete all entries from the telephone number list

Press the ▼ button. → The last call will be shown at first entry in the call list.

→ Navigate through the entries using the keys ▲ ▼.

→ Press **Option** (left softkey).

→ **Delete all** → Press **OK** (left softkey).


→  appears on the display.

→ Press **OK** (left softkey) to delete all entries.

Various telephone number lists

The »freeTEL III« has a call list for incoming and outgoing calls.

In these lists incoming and outgoing numbers are saved that can subsequently be saved in the telephone directory or deleted from the list.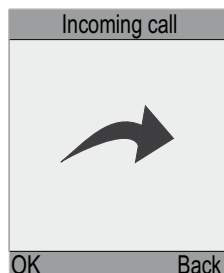


Telephone number list of incoming calls

Press **Menu** (left softkey). → **Incoming call**

- Press **OK** (left softkey).
- Navigate through the entries using the keys **▲▼**.
- Press **Option** (left softkey) to get to the menu for more functions (**Store**, **Delete** or **Delete all**).

or:

- Press **Back** (right softkey) to return to the menu list.

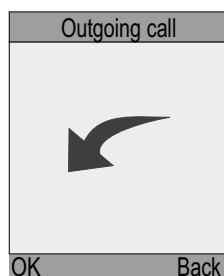Telephone number list of outgoing calls

Press **Menu** (left softkey) → **Outgoing call**

- Press **OK** (left softkey).
- Navigate through the entries using the keys **▲▼**.
- Press **Option** (left softkey) to get to the menu for more functions (**Store**, **Delete** or **Delete all**).

or:

- Press **Back** (right softkey) to return to the menu list.

Answering machine settings (TAM)

Recording an outgoing message

Attention:

Please speak loudly and clearly into the microphone when recording a message.

Press **Menu** (left softkey) → **TAM**

→ Press **OK** (left softkey)

→ **OGM** → Press **OK** (left softkey)

→ Select desired message (**OGM 1** or **OGM 2**) using the keys **▲ ▼**.

→ Press **OK** (left softkey) → **Record**

→ Press **OK** (left softkey)

→ The time and „Recording..“ are shown on the display and you can record your message now (min. 3 seconds).

Press **Stop** (left softkey) to stop recording. After recording the message is played for checking.

Note:

The recording is automatically terminated once the maximum recording duration of 2 minutes is reached.

Note:

The 1st outgoing message is only played in “Answer/Record“ mode, and the 2nd outgoing message is only played in “OGM only“ mode.

Listening to an outgoing message OGM

Press **Menu** (left softkey) → **TAM**

→ Press **OK** (left softkey)

→ **OGM** → Press **OK** (left softkey)

→ Select desired message (**OGM 1** or **OGM 2**) using the keys **▲ ▼**.

→ Press **OK** (left softkey) → **Play recording**

→ Press **OK** (left softkey)

→ “Playback..“ is shown on the display and you can listen to the recorded message.

Press **Stop** (right softkey) to terminate the playback early.

Note:

If you would like to listen to the message via the loudspeaker, press the left softkey during playback.

Switching on the answering machine (TAM)

Please note: An outgoing message must be recorded on the machine before you can make one of these settings.

You have a choice of 6 different settings:

- | | |
|---------------|--|
| Answer off | The answering machine is switched off. The telephone doesn't answer calls. |
| Announce Only | The telephone only plays an outgoing message when it receives a call. The caller is not able to leave a message. |
| Answer/2-ring | The answering machine switches on after 2 rings and is ready to record information. |
| Answer/4-ring | The answering machine switches on after 4 rings. |
| Answer/6-ring | The answering machine switches on after 6 rings. |
| Answer/8-ring | The answering machine switches on after 6 rings. |

Procedure for defining the "Answer/2-ring" setting, for example:

- Press **Menu** (left softkey) → **TAM**
- Press **OK** (left softkey)
- **Ans/Ring** → Press **OK** (left softkey)
- Select desired function ("Answer/2-ring") using the keys ▲ ▼.
- Press **OK** (left softkey) to save your settings.

Set remote query PIN

- Press **Menu** (left softkey) → **TAM**
- Press **OK** (left softkey)
- **Remote PIN** → Press **OK** (left softkey)
-  Enter the desired PIN (max. 4 digits, other than 0000).
- **OK** Press **OK** (left softkey)
-  Re-enter the desired PIN (max. 4 digits).
- Press **OK** (left softkey) to save the PIN entry.

Messages

General

The handset flashes white and „New msgs“ is shown on the display if new messages were recorded.

Playing a message

Press ▲ button to select the menu “Message” → **Play recording**

→ Press **OK** (left softkey)

→ The number of new messages and the number of saved messages is shown.

→ Messages are played back.

Note:

If you would like to listen to the messages via the device loudspeaker, press **SPK** (left softkey) during playback.

If you would like to delete a message during playback, press **Clear** (right softkey).

Note:

For better separation of the recorded messages in new and old messages “New message” or “Old message” is shown on the display.

Deleting all old messages

Press ▲ button to select the menu “Message” → **Delete old**

→ Press **OK** (left softkey).

→ **?** appears on the display.

→ Press **OK** (left softkey) to delete the messages.

Recording a memo

You can record a memo which is played as a normal message.

Press ▲ button to select the menu “Message” → **Record memo**

→ Press **OK** (left softkey).

→ The time and „Recording..“ are shown on the display and you can record your message now (min. 3 seconds).

Press **Stop** (left softkey) to terminate the playback.

Answering machine on the go (remote control)

General

You can retrieve messages from the answering machine from any other phone (e.g. hotel, telephone booth).

The following conditions must be met:

1. You have set a remote query PIN other than 0000 (refer to set remote query PIN, page: 20)
2. The phone used for the remote query is tone dial enabled (DTMF), i.e. when pressing a key you hear different tones.

Start remote control

Dial your own number.

“PIN” Once you hear your message text: enter remote query PIN via the keypad.

→ You will hear a signal tone to confirm your entry.

Now you can operate the answering machine via the keypad of the other telephone.

Note:

If you have entered the wrong remote query PIN, the call is terminated.

Listen to messages and manage

Play back recorded messages

→ Press key

For pause

→ Press key

For re-recording

→ Press key again

Repeat current message

→ Press key

Repeat last played message

→ Press key 2x

Skip current message

→ Press key

Delete current message

→ Press key

Terminate playback of current message

→ Press key

Listen to announcement 1 or record again

Play message 1 → Press key **8^{TV}**, then key **1**
Terminate playback of message 1 → Press key **6^{MO}**

Record message 1 → Press key **7^{RT}**, then key **1**
Terminate recording and play message 1 → Press key **6^{MO}**

Listen to announcement 2 or record again

Play message 2 → Press key **8^{TV}**, then key **2^{MC}**
Terminate playback of message 2 → Press key **6^{MO}**

Record message 2 → Press key **7^{RT}**, then key **2^{MC}**
Terminate recording and play message 2 → Press key **6^{MO}**

Recording a memo

Record memo → Press key **5^{ML}**
Terminate recording → Press key **6^{MO}**

Set answering machine mode

Set "Answer/2-ring" → Press key **9^{RF}**, then key **2^{MC}**

Set "Answer/4-ring" → Press key **9^{RF}**, then key **4^{MR}**

Set "Answer/6-ring" → Press key **9^{RF}**, then key **6^{MO}**

Set "Answer/8-ring" → Press key **9^{RF}**, then key **8^{TV}**

Set "Announce Only" → Press key **9^{RF}**, then key **3^{MR}**

Set "Answer off" → Press key **9^{RF}**, then key **5^{ML}**

Note:

The deactivated answering machine can be activated via remote control as follows:

-  Dial your own number.
- Let the phone ring for at least 15 times.
- Enter the remote query PIN via the keypad.
- You will hear a signal tone to confirm your entry.

Now you can operate the answering machine via the keypad of the other telephone (see above).

Melody/Handset ringer volume

The handset's melody/ringer volume can be set as follows:

Press **Menu** (left softkey) → **Ringer**

→ Press **OK** (left softkey)

→ Use the ◀ and ▶ buttons to select the desired melody.

→ Press the ▼ button.


→ Use the ◀ and ▶ buttons to select the desired ringer volume.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Setup

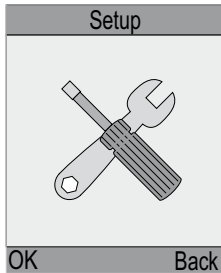Changing the menu language

You can set the display texts to different languages.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Language**

→ Press **OK** (left softkey)

→ Use the ◀ and ▶ buttons to select the desired language.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Automatic call acceptance

If the function is activated simply take the handset from the base station without having to press the ☎.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **Auto answer**

→ Press **OK** (left softkey)

→ Use the ◀ and ▶ buttons to select "ON" or „OFF“

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Set contrast

You can adjust the display contrast in three shades for better legibility of the texts according to your own wishes.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **LCD contrast**

→ Press **OK** (left softkey)

→ Use the ◀ and ▶ buttons to select one of the three possible shades.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Activate/deactivate key tones

Your phone can signal each key activation acoustically

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **Key tone**

→ Press **OK** (left softkey)

→ Use the ◀ and ▶ buttons to select "ON" or „OFF“

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Changing the handset name

You can change the "Handset 1" text on your handset to your own personal text.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **Handset name**

→ Press **OK** (left softkey)

→ Delete the characters left of the cursor using **Entf.**.

→ Enter the required name with the buttons of the keypad.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Note:

In addition to the name, a number between 1 and 6 is displayed, depending on the number of handsets added.

Change menu colour

You can select in which colour the menu bar and the text bar of the softkeys will be marked.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **Menu color**

→ Press **OK** (left softkey)

→ Use the ▲▼ or ◀▶ buttons to select the desired colour.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Activate/deactivate time

You can select if the current time is to be shown when the handset is in the base station.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **Clock**

→ Press **OK** (left softkey)

→ Use the ◀▶ buttons to select "ON" or "OFF"

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Change background colour of splash screen

You can select which background colour the splash screen should have.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **Wallpaper**

→ Press **OK** (left softkey)

→ Use the ▲▼ or ◀▶ buttons to select the desired colour.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Change background colour of menu lists

You can select which background colour the menu list should have.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Handset**

→ Press **OK** (left softkey) → **Background**

→ Press **OK** (left softkey)

→ Use the ▲ ▼ or ◀ ▶ buttons to select the desired colour.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Register a handset

If more than one handset is used, they must be registered on the base station first. Each additional handset must be registered separately. Up to six handsets can be registered on any one base station.

Note:

The handset supplied with the base station is already registered. You don't need to register this handset.

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Registration**

→ Press **OK** (left softkey) → **Register new**

→ Press **OK** (left softkey)

→ **?** appears on the display.

→ Press **OK** (left softkey)

→ Enter the RC code (**1111**)

→ Press **OK** (left softkey). "Please wait" appears on the display.

→ Press registration/paging **●|||** button at the base station for five seconds and release the button again.

The LED top right at the base station blinks during the search mode of the base station.

→ After correct registration you can hear a signal tone and the handset returns to the registration menu.

When returning to the start menu, you should be able to see the **Y** symbol in the display.

The handset is now correctly registered.

Note:

Press **Back** (right softkey) to return to the menu list.

Register other handsets

Mobile sets of other GAP enabled devices can be registered as follows:

At the mobile set: Start the registration of the mobile set following the respective user manual.

At the base: Press registration/paging  button at the base station for five seconds and release the button again. The LED top right at the base station blinks during the search mode of the base station.

If you are asked to enter a RC code, kindly enter “**1111**” as registration code.

Deleting a handset from the base station

Should you wish to deregister a mobile set from the base station, proceed as follows:

Press  (left softkey) → **Setup**

→ Press  (left softkey) → **Registration**

→ Press  (left softkey) → **Delete**

→ Press  (left softkey)

→  appears on the display.

→ Press  (left softkey)

→ After a short while the handset returns to the start menu and is now de-registered from the base.

Dial pause setup

In order to get an exchange line on a telephone system, you can insert a dial pause after the first digit. The pause can be set between 1 and 6 seconds.

Press  (left softkey) → **Setup**

→ Press  (left softkey) → **Network**

→ Press  (left softkey) → **Pause**

→ Press  (left softkey)

→ Use the   buttons to select the desired dial pause (1 to 6 seconds).

→ Press  (left softkey) to save your settings.

Note:

Press  (right softkey) to return to the menu list.

Flash time setup (Inquiry button)

This function is required in order to activate certain T-Net features. If you operate your device on a telephone system, you may have to change the flash time (see the operating manual for the telephone system).

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Network**

→ Press **OK** (left softkey) → **Recall**

→ Press **OK** (left softkey)

→ Use the **▲** **▼** buttons to select the desired flash time.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Note:

The flash time can be set to the following times: 90/270/600 ms. Fixed setting standard 90 ms.

Dial mode setup

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Network**

→ Press **OK** (left softkey) → **Dial mode**

→ Press **OK** (left softkey)

→ Use the **▲** **▼** buttons to select the desired dial mode (Tone or Pulse)

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Note:

Nowadays, most telephone lines use tone dialing.

Key lock

To prevent that you might accidentally make a call, you can activate a key lock. When activated, all key functions are disabled and therefore the phone is protected from accidental use e.g. by children. To activate the key lock, proceed as follows:

1. Press down and hold the  button until the  icon appears in the display.
2. To reactivate the keys, press down and hold the  button until the  icon disappears in the display. The key pad will be activated once again.

Disabling the permanent RF radiation

The base station and the hand set continuously exchange information in standby mode and during calls via radio waves that in case of an incoming call a ring tone and the caller ID can be transmitted from the base station to the mobile set. These radio waves can be reduced or deactivated completely if the handset is close to or inside the charger of the base station.

In order to activate one of the 5 functions proceed as follows:

Press **Menu** (left softkey) → **Setup**

→ Press **OK** (left softkey) → **Low radiation**

→ Press **OK** (left softkey)

→ Use the **▲ ▼** buttons to select the desired function.

→ Press **OK** (left softkey) to save your settings.

Note:

Press **Back** (right softkey) to return to the menu list.

Possible functions and their operation:

Minimum - The base station and the handset exchange information only during a call. In standby mode the radio waves are almost completely deactivated.

Very low - The base station and the handset exchange information during a call and within limits in standby mode. In addition the strength of the radio waves is strongly reduced.


Low - The base station and the handset exchange information during a call and to some extent during standby mode. In addition the strength of the radio waves is strongly reduced.

Normal - The base station and the handset continuously exchange information. There is no reduction of the radio waves.

Automatic - The base station and the handset exchange information during a call and in standby mode. In addition the strength of the radio waves is automatically adjusted depending on the distance between base station and handset.

Kindly keep in mind that the range for the settings Minimum, Very low and Low is less due to the reduced transmitting power. If you have selected one of the previously mentioned settings you can phone only in the immediate vicinity of the base station.

Use several handsets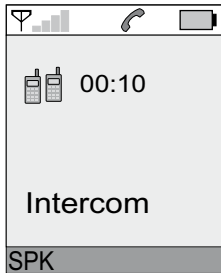

Internal calls

If there are several handsets registered to a base station, internal calls can be made between these telephones.

“**INT**” Press **▶** button

- **☰** Enter the required handset number.
- The desired handset rings
- Press **📞** button to answer the call.
- End the call by pressing the **📞** button.

Transferring calls between handsets

An external call can be connected from one handset to another (both handsets have to be registered to the same base station).

You have received an incoming call and would like to transfer that call to another one of the cordless phones

→ “**INT**” Press **▶** button

→ **☰** Enter the required telephone number.


→ The desired handset rings

→ Press **📞** button to answer the call at the called handset.

→ Press **📞** button on the handset on which the call is being made once the internal user answers.

Bluetooth®

(This function is only available for a freeTEL III with article number A-4605..)

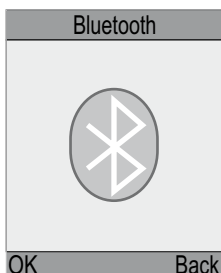Activate/deactivate Bluetooth® function

Press **Menu** (left softkey) → **Bluetooth**

→ Press **OK** (left softkey) → **BT Mode**

→ Press **OK** (left softkey)

→ Use the **◀** and **▶** buttons to select “ON” or “OFF”

→ Press **OK** (left softkey) to save your settings.

Note:

If the Bluetooth® function is activated, the batteries discharge quicker than during normal operation. Deactivate the function if you do not need it!

Couple Bluetooth® devices

You have the possibility to connect your »freeTEL III« via the Bluetooth® connection with a Bluetooth® headset or other Bluetooth® enabled devices.

Press **Menu** (left softkey) → **Bluetooth**

→ Press **OK** (left softkey) → **BT Pairing**

→ Press **OK** (left softkey)

→ Activate your Bluetooth® enabled device and set it in coupling mode (see user manual of the Bluetooth® enabled device)

→ If the connection between »freeTEL III« and Bluetooth® enabled device was established successfully, ✓ will be shown on the display.

Note:

The Bluetooth® function has to be activated that other Bluetooth® enabled devices can be connected to your »freeTEL III«.

Connecting special accessories

The »freeTEL III« possesses:

- Standard headset port with a 2.5 mm socket (for connecting a headset, for instance).
- Standard headphone port with a 3.5 mm socket (for connecting a headphone, for instance).

The following special accessories can be connected to the base station:

- Vibrating pillow
 - Switch module
 - Flash module
 - Acoustic module
-

Troubleshooting

General

- Make sure that the telephone cable is undamaged and correctly inserted. Disconnect any other accessories – extension cable and other telephones. If the telephone works afterwards, this indicates that the fault is caused by another device.
- Make sure that the power cord is attached to the base station and plugged in correctly.
- Check if the batteries are charged.
- Try connecting the telephone in a neighbour's house, for example. If the device works there, this indicates that your phone line may be faulty. Contact your telephone providers' help line.

- Connect another telephone to your line. If it works, this would mean that your telephone is probably faulty.

Warning tone during a call

- The batteries are virtually empty.
 - Place the handset onto the charger.
- The telephone's range has been exceeded.
 - Take the handset closer to the base station.

Telephone number is not displayed

- The caller display must be supported by your telephone operator.
- The caller display function doesn't work if the telephone is connected to an extension.

Resetting the telephone to the factory settings

Procedure for resetting the telephone to the ex-factory status:

1. Press **Menu** (left softkey).
2. Now use the keypad to enter the following code: "4,6,1,5,3,0,5".
3. **?** appears on the display.
4. Press **OK** (left softkey). "Please wait" appears on the display.
5. After about 2 seconds, **✓** appears on the display and the »freeTEL III« returns to the start menu. Now the telephone has been reset to the default settings.

Important Notice

This product is **not** protected against splash water. Do not place any containers filled with water, such as flower vases on or near the product. Also, do not place anything with an open flame near it.

Appendix

Maintenance and care

The telephone is maintenance-free. If it becomes dirty, give it an occasional wipe with a soft, damp cloth. Never use alcohol, thinner or any other organic solvents.

The telephone should not be exposed to direct sunlight for a long period of time. Furthermore, it should be protected from high levels of heat, moisture or strong mechanical vibrations.

Warranty

The telephone operates very reliably. However, should malfunctions occur despite it being installed and used correctly, please contact your local dealer or contact the manufacturer directly. The warranty includes free repair and return. The requirement for this is that it must be sent in its original packaging. Therefore, do not throw this away. The warranty will become void in the event of damage resulting from incorrect handling or repair attempts by unauthorized people (breaking the seal on the device). Repairs will only be carried out under warranty if the completed warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

Quote the serial number of the device in all correspondence.

Environmental note/Waste disposal

Disposal of used electrical and electronic equipment (applicable in the countries of the European Union and in other European countries which operate a separate collection system for these devices). The symbol on the product or its packaging indicates that this product should not be treated as normal household waste. It should be handed in to a collection center for recycling electrical and electronic equipment. By playing your part in ensuring the correct disposal of this product, you will be safeguarding the environment and the health of other people. The environment and health are jeopardized by incorrect disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain further information about recycling this product from your local council, from local waste disposal operators and from the shop where you bought the product.

Rechargeable batteries

The rechargeable batteries that come with the device can be recycled. Please dispose of these rechargeable batteries in receptacles designed for that purpose or return them to a retail outlet. Dispose only of those batteries that have been completely discharged to ensure environmental protection.

Technical Data

Standard:	DECT = Digital Enhanced Cordless Telecommunication Advance digital standard for cordless telephones GAP = Generic Access Profile Transmission protocol for cordless telephones
Number of channels:	120 duplex channels
Transmission frequency range:	1880 MHz to 1900 MHz
Duplex method:	Multiplex time division
Channel bandwidth:	1728 KHz
Bit rate:	1152 Kbit/s
Modulation:	GFSK
Transmission power:	10 mW, average power per channel
Range:	up to 50 m indoors up to 300 m outdoors
Power supply:	Base station: Primary: 100-240V AC 50/60Hz 150mA Secondary: 7,2V DC / 500 mA Handset: NiMH, AAA size, 1.2V
Power consumption:	approx. 7 W
Service life:	Standby: up to 100 hrs Talk time: up to 10 hrs
Dimensions (WxHxD):	Base station: 116 x 138 x 86 mm Handset: 178 x 51 x 33 mm
Weight:	Base station: approx. 224 g Handset: approx. 175 g (with batteries)

CE symbol

This telephone satisfies the following EU directives:

- 2011/65/EG RoHS directive
- 2004/108/EG EMC directive
- 2002/96/EG WEEE directive
- 2006/95/EG Low voltage directive
- 1999/5/EG R&TTE directive

Compliance with the directives listed above is confirmed by the CE seal on the telephone. CE compliance declarations are available on the Internet at www.humantechnik.com.

Technical specifications subject to change without notice.

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Sarabec Ltd