freeTEL *eco*

D

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Seite 2

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Registering a new handset

Up to 5 base stations can be registered on a base station. You can then also make internal telephone calls between the handsets. You can also forward incoming calls to other handsets.

You can register up to 4 base stations on each handset. This increases the range of the telephone so that you can move between the telephone transmission areas.

To register a new handset, proceed as follows:

1. Press and hold the „MT-Search“-button , located at the underside of the base, for approximately 5 seconds.
2. Press the key “Menu” and select the entry “Registration” with .
3. Now press “Select”.
4. With  select the preferred base and confirm with “Select”. The base station is now located and the handset is registered.
5. Now enter the PIN code (default is 0000).
6. If the PIN code is accepted, a log-on tone sounds and the base assigns a number to the handset. If the PIN code is incorrect, a warning tone sounds and the handset returns to the previous registration status. You can re-enter the PIN code.

Internal calls

If you have registered several handsets to a base, you can hold internal calls, and connect external calls between the handsets.

To make an internal call, proceed as follows:

1. Press the  key on the handset.
2. Enter the desired handset number from 1 to 5. With 9, all handsets are called.
If you enter a wrong number, the handset will give a warning tone.

When you receive an internal call, the number of the calling handset will be shown on the display.

Connecting calls between handsets

An external call can be forwarded from one handset to another.

Proceed as follows:

1. Press the  key on the handset during the call. The external caller is put on hold.
2. Enter the desired handset number from 1 to 5.
3. The dialled handset will ring. The number of the calling handset will be shown on the display.
4. Answer the call with .
5. If you like, you can now speak with the selected user. You can now hang up before the user accepts the call, and the call is then directly forwarded.
6. Press the  key to end the call. The caller on hold is automatically forwarded to the other caller.

To cancel the transfer, press the  key again and you can speak with the external caller again.

If the selected handset is not answered within a certain time, the call is reset.

Important note

The device is **not** protected against splashing water. Do not place objects that are filled with liquid, such as vases, on the device. Likewise, no open flames, such as lit candles, may be placed on the device.

Appendix

Maintenance and care

The telephone is maintenance-free. If it becomes dirty, you should occasionally clean the device with a soft, damp cloth. Never use alcohol, paint thinner or other organic solvents. The telephone should not be exposed to direct sunlight for long periods of time, and it should especially be protected from excessive heat, moisture and strong mechanical shocks.

Maintenance

The telephone has a high degree of operating reliability. Should problems occur, despite proper installation and operation, please contact your dealer or the manufacturer directly. The warranty covers free repair and free return delivery. This requires shipping in the original packaging, so do not throw this out. The warranty is void in the case of damage which was caused by improper handling or attempted repair by unauthorised persons (destruction of the seal on the unit).

Warranty repairs are only carried out when the completed warranty card is sent with a copy of the invoice/receipt of the dealer.

The device number must always be indicated.

Note on the environment / disposal

Disposal of waste electrical and electronic equipment (applicable in the countries of the European Union and other European countries with a separate collection system for these devices). The symbol on the product or its packaging indicates that this product is not to be treated as normal household waste, but that it must be returned to a collection point for the recycling of electrical and electronic equipment. By ensuring this product is disposed of correctly, you protect the environment and the health of those around you. The environment and health are jeopardised through improper disposal. Material recycling helps to reduce the consumption of raw materials. You can obtain more information about recycling this product from your local authority, public waste disposal service or the shop where you purchased the product.

Batteries

The supplied batteries are recyclable. Please dispose of batteries via the battery container or the dealer. To guarantee environmental protection, only dispose of discharged batteries.

Technical details handset

Standard:	DECT = Digital Enhanced Cordless Telecommunication Extended digital standard for cordless telephones Transmission protocol for cordless telephones
Number of channels:	120 duplex channels
Transmission frequency range:	1880 MHz to 1900 MHz
Duplex methods:	Multiplex time division
Channel bandwidth:	1728 KHz
Bit rate:	1152 Kbit/s
Modulation:	GFSK
Transmission performance:	10 mW, medium performance per channel
Range Alarm:	up to 50 m indoors up to 300 m outdoors
Power supply:	Charging station: Primary: 100-240V AC 50/60Hz Secondary: 7.5V DC / 500 mA Handset: NiMH batteries, size AAA, 1.2V, 800 mAh
Operating time:	Standby: up to 180 hours Talk time: up to 10 hours
Dimensions (HxWxD):	Charging station: 100 x 155 x 55 mm Handset: 185 x 57 x 34 mm
Weight:	Charging station: approx. 148 g Handset: approx. 174 g (with batteries)

CE marking

This device fulfils the requirements of the following EU Directives:

- 2002/95/EC RoHS Directive
- 2004/108/EC EMC Directive
- 2002/96/EC WEEE Directive
- 2006/95/EC Low Voltage Directive
- 1999/5/EC R & TTE Directive

Conformity to the above directives is indicated by the CE marking on the device.
EC Declarations of Conformity are available online under www.humantechnik.com.

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Sarabec Ltd