HUMANTECHNIK

**D**

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Seite 2

**GB**

**Operating Instructions**

Bone conduction receiver

Page 6

**F**

**Mode d'emploi**

Combiné ossivibrateur

Page 10

**NL**

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Beengeleidingshoorn

Pagina 14

**I**

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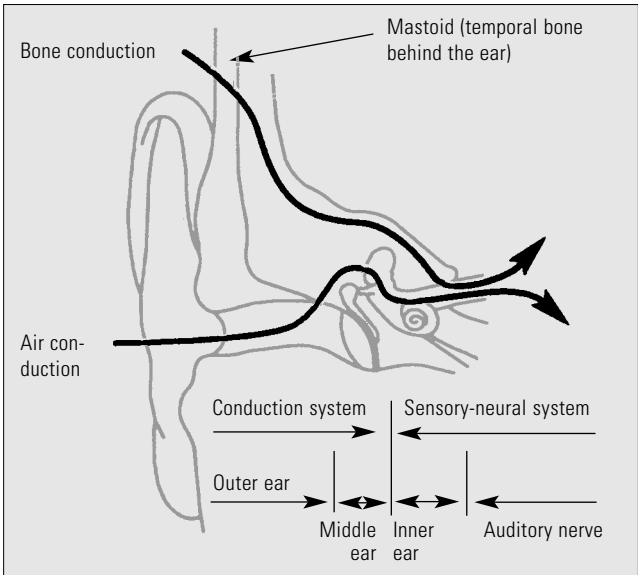
## Bone Conduction Receiver

We would like to congratulate you on the purchase of our *bone conduction receiver* and hope that it will serve you well for a long time to come. You have chosen a modern and reliable system. Please read the operating instruction through carefully to be able to start the unit correctly and to familiarize yourself with all of the system features.

### **Standard components**

Please check if all following components are included:

- *Bone conduction receiver*
- Operating instruction
- Warranty card

If any parts are missing please immediately contact your dealer or the manufacturer directly.

### **How it works**

Sound and noise usually take two paths via different transmission chains from the auditory nerve to the brain. The most well-known way is via air conduction, whereby the sound enters through the outer ear and travels through the inner ear to the auditory nerve. The second, less well-known way is via bone conduction. In this case, the sound is transmitted through the head bone to the auditory nerve. Thus, the *bone conduction receiver* complements and supports the conventional method of sound transmission through vibration.

When the *bone conduction receiver* is affixed in the vicinity of the outer ear, make sure that it makes good, secure contact to the head bone. The sound is transmitted by vibration through the head bone directly to the auditory center (see illustrations).

The *bone conduction receiver* can be connected to the following devices:

- *CM-1* and *CM-light* communication systems
- *Crescendo 50* sound amplifier
- *CM-BT* Bluetooth amplifier
- *TA-2* and *PL-51* telephone amplifiers
- *flashTEL eco*, *flashTEL comfort II* and *flashTEL comfort II f* telephones
- *LR-Version* TV amplification systems

The receiver can also be connected to other devices equipped with an audio output jack (3.5 mm), e.g. an MP3 player, CD player, conventional telephones.

### **Maintenance and care**

The *bone conduction receiver* is virtually maintenance-free. Should it become soiled, clean the device occasionally with a cleaning cloth from our Hygienic Set (A-4982-0) or using any other soft, damp cloth. Never use alcohol, thinner or any other organic solvent.

Do not expose the bone conduction receiver to direct sunlight for longer periods of time and protect it against extreme heat, extreme dust, moisture and strong mechanical vibration.

**Important:** The device is **not** protected against splash water. Avoid any contact of the receiver with moisture and heat sources.

### Warranty

The *bone conduction receiver* is very safe to operate and provides highly reliable service. Should a malfunction occur despite the unit having been set up and operated correctly, please contact your dealer or the manufacturer directly.

This warranty covers the repair of the product and returning it to you free of charge. It is essential that you send in the product in its original packaging, so do not throw the packaging away.

The warranty does not apply to damage caused by incorrect handling or attempts to repair the unit by people not authorised to do so (destruction of the seal on the unit). Repairs will not only be carried out under warranty if the complete warranty card is returned accompanied by a copy of the dealer's invoice/till receipt.

**Always specify the product number in any event.**

**Disposal** of used electric and electronic units (applicable in the countries of the

European Union and other European countries with a separate collection system).

The symbol on the product or the packaging indicates that this product is not to be handled as ordinary household waste but has to be returned to a collection point for the recycling of electric and electronic units. You protect the environment and health of your fellow men by the correct disposal of this products. Environment and health are endangered by a faulty disposal. Material recycling helps to reduce the consumption of raw material. You will receive further information on the recycling of this product from your local community, your communal disposal company or your local dealer.

## Specifications

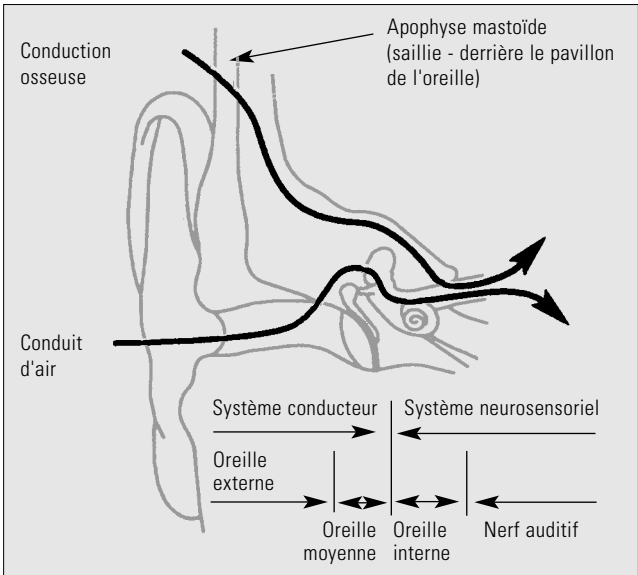
Weight:	35 g
Power input:	30-100 mW
Amplification:	Max. 88 dB
Cord:	120 cm, jack ø 3.5 mm stereo

This product complies with European Union norms.  
You can obtain EC compliance declarations from your dealer, or directly from the manufacturer of this product.

Technical specifications subject to change without notice.

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## **Humantechnik GHG AG**