**DE**

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Seite 2

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**EN**

**Instruction manual**

Page 8

**Speech transfer system**

**FR**

**Mode d'emploi**

Page 14

**Système de transfert de la parole**

**NL**

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Pagina 20

**Spraakoverdrachtsysteem**

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
## Package contents

Please check if the following parts are included:

- Microphone base
- Microphone neck
- Foam head for microphone
- External communication unit
- 1 connector block
- 3 x cable fixings
- 3 x cable straps
- Strain relief
- Adhesive strip for strain relief
- User manual
- Warranty card

If any parts are missing, please contact AUDIOropa directly.


## Above view

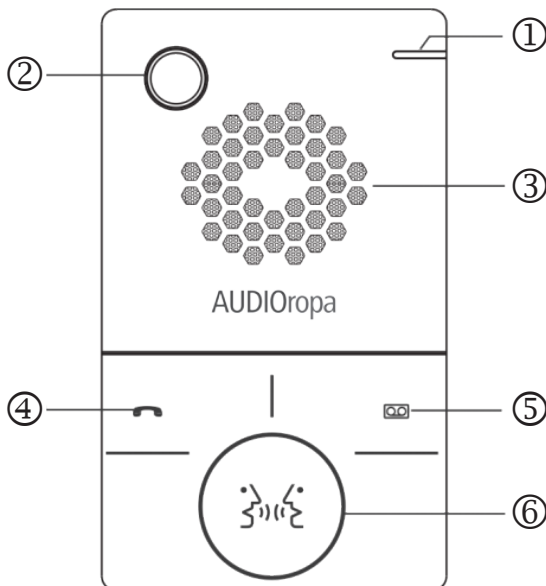
- ① Status indicator. This lights up red when the unit is ready for operation.
- ② Microphone socket: Insert the microphone neck and tighten the knurled screw to fix it in place.
- ③ Loudspeaker: Outputs the transmission of the external communication unit.
- ④ Announcement button (left): When the intercom button ⑥ lights up blue: Briefly press the announcement key to play the announcement »Welcome. How can I help you?« via the external communication unit.  
Press the announcement button for 1 second to play the announcement »Good bye«. The intercom button flashes green during the announcements.
- ⑤ Without relevant function.
- ⑥ Intercom button: Press the button to start voice transmission via the external communication unit. Press the button again to end the transmission. During an active transmission, the intercom button lights up blue.

## Side view

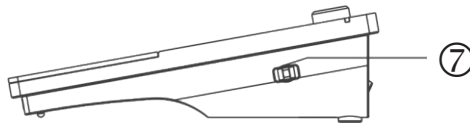
- ⑦ Level switch. There are 3 level settings available:  
Switch to front: Loud  
Middle position: Standard  
Switch to rear: Soft  
The standard setting is used by default.

## Connection cable

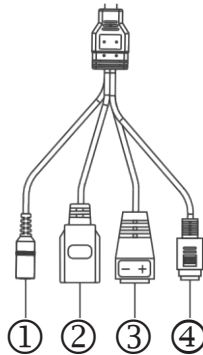
- ① Power supply: For connection with the power supply unit.
- ② RJ45-connector: Without relevant function.
- ③ Audio output: For connection to loop amplifiers, e.g. AUDIORopa *LA-70* or other independent audio systems. Use the included connector block for this purpose.
- ④ Connector for the external communication unit.

## External communication unit

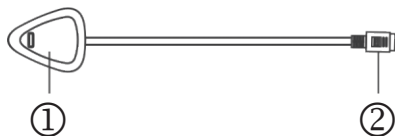
- ① Communication unit: Serves as loudspeaker and microphone for speech transmission to the microphone base.
- ② Connector to the microphone base.

## Connection diagramm
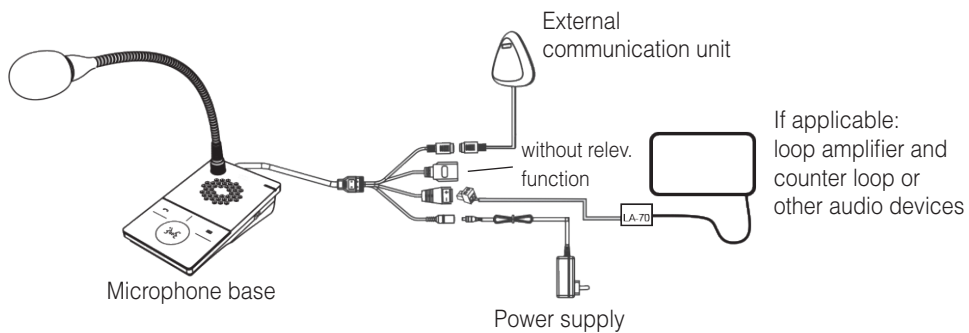
## Dimensions, in mm
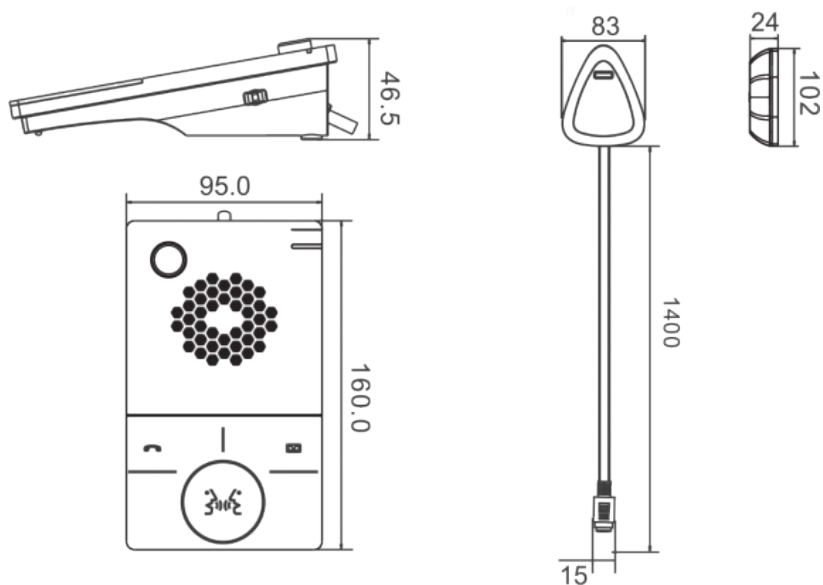
## Installation instructions

1. Place the microphone base in a suitable location.
2. Attach the external communication unit to a glass pane or other smooth and clean surface. To do this, peel off the protective paper from the adhesive surface and stick the communication unit in the desired location.
3. Lay the cable using the cable holders and cable ties. Stick the self-adhesive cable fixings in the desired place and fasten the cable to the cable holders with the cable ties.
4. If necessary, the strain relief can be placed over the cable and fixed with the included double-sided adhesive tape strip.

## Specifications

Power supply:	DC 12 V / 1,5 A
Power consumption:	≤ 5 W
Connectors:	Power supply, analogue audio output, network connector (without function), PS/2 connector for external communication unit
Signal-to-noise ratio:	>90dB
Frequency response:	20 Hz - 16 KHz
Audio sampling:	8 KHz - 32 KHz
Temperature range:	-10°C - 55°C
Humidity range:	≤ 90% RH (non-condensing)
Dimensions / weight:	Microphone base 95 x 160 x 46.5 mm / 0.44 kg External unit 83 x 102 x 24 mm / 0.2 kg

This device complies with the following EC directives:

- 2011 / 65 / EC + 2015 / 863 / EC RoHS-directive
- 2012 / 19 / EC WEEE-directive
- 2014 / 30 / EC EMC-directive
- 2014 / 35 / EC LVD-directive

Compliance with the directives listed above is confirmed by the CE seal on the device.  
CE compliance declarations are available on the Internet at  
[www.audioropa.com](http://www.audioropa.com).

Technical specifications subject to change without prior notice.

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**Sarabec Ltd**